There is no substitute for quality

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**Dentistry / Dental Technique
Odontologie / Prothese Dentaire
Odontologia / Protesis Dental**

**Polierer und Schleifkörper
Polishers and abrasives
Polissoirs et abrasifs
Pulidores y abrasivos**

2017- 2019

There is no substitute for quality

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„There is no substitute for quality“ this conviction has committed our company's philosophy since our foundation in 1905.

Our ideas and all processes of research & development, production and distribution have to meet with the demands of this philosophy.

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Universal pre-polishers, for all restoration materials

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2-step diamond interspersed polishing system for composite (also high filled)

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Polishing, pink beige

Diamond, PUR
Grit size medium
Bond hardness medium

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2-step polishing system for composite

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Polishing, yellow

High-grade corundum Silic. bond
Grit size medium
Bond hardness soft

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Also suitable for the removal of
adhesive residues after bracket
treatment.

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End-polishing, blue

High-grade corundum,
Silicone bond
Grit size fine
Bond hardness soft

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2-step polishing system for precious and non-precious metal alloys, amalgam

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Polishing, brown

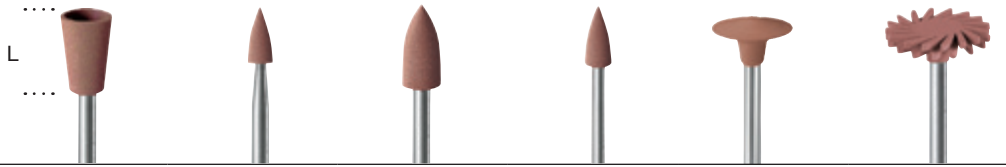
SiC, PUR
Grit size medium
Bond hardness medium

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End-polishing, green

SiC, PUR
Grit size fine
Bond hardness soft

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2-step diamond interspersed polishing system for ceramics / silicate ceramics

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End-polishing, green yellow

Diamant, PUR
Grit size fine
Bond hardness soft

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2-step diamond interspersed polishing system for zirconium oxide

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Polishers and brushes for prophylaxis

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for the application with polishing paste, white

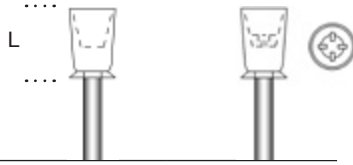
Silicone polisher
(without abrasive grain)
Bond hardness medium

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BIMS PUR,
Korngröße extra fein

for the application without polishing paste, white, green, yellow

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for the application with polishing paste, white

Polyamide brushes
(without abrasive grit)

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for ceramic, acrylics, metal alloys, enamel

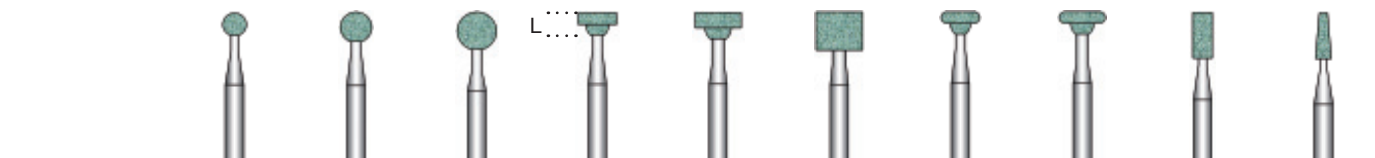
Silicon carbide abrasives
(SiC), green
Ceramic bond
Grit size medium
Bonding hardness medium

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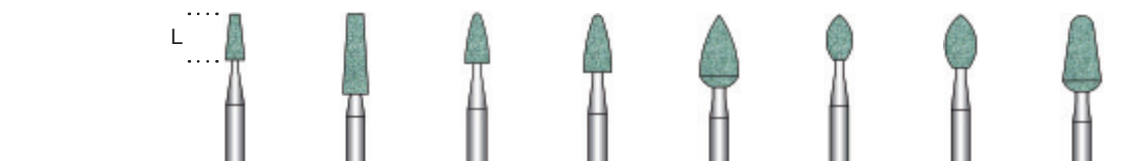
for smooth grinding of composites

High-grade corundum
Arkansas abrasives (Al₂O₃), white
Ceramic bond
Grit size extra fine
Bonding hardness hard

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Universal pre-polishers for all materials

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Smoothing, white

SiC, Silicone
Grit size coarse
Bond hardness soft

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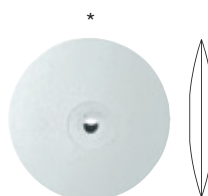
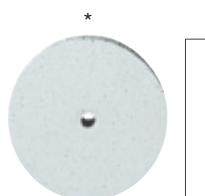
Pre-polishing for model casting, CrCo, mocha

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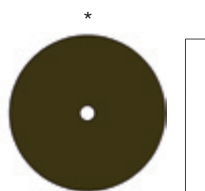
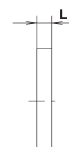
2-step polishing system for precious metal alloys

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SOFT WHEELS; impregnated buffs for metals

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Impregnated cotton buffs (application without polishing paste)

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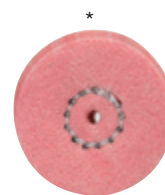
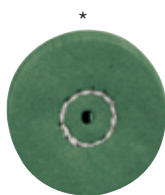
Cambric, fine nettle cloth for high-lustre polishing with diamond paste.

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Chamois, for high-lustre polishing after the use of paste.

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Cotton, for the final high-lustre polishing after the use of paste.

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Microfiber for the final high-lustre polishing after using paste.

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for ceramic, acrylics, metal alloys

Silicon carbide abrasives (SiC), green

Ceramic bond
Grit size medium
Bonding hardness medium

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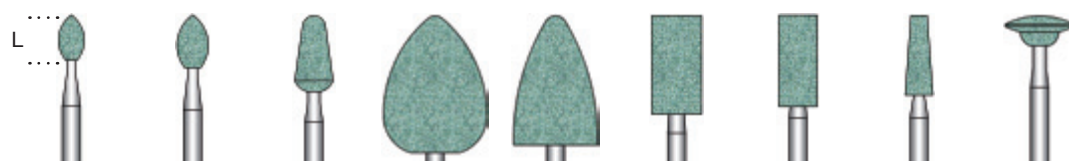
for ceramics

Silicon carbide abrasives (SiC), green

Ceramic bond
Grit size fine
Bonding hardness medium

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for ceramics, acrylics, metal alloys

Silicon carbide abrasive wheels (SiC), green

Ceramic bond
Unmounted,
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness medium

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for precious metal alloys and ceramics

**Silicon carbide
Separating discs (SiC), green**

Resin bond
Unmounted
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness hard

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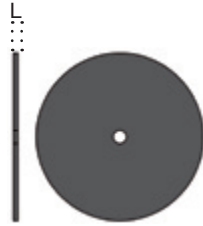
for acrylics and metal alloys

**Silicon carbide
abrasive wheels (SiC),
green**

Ceramic bond
Unmounted
Ø of centre hole 6,5 mm
Thickness 6,5 mm
Grit size medium, coarse
Bonding hardness
medium

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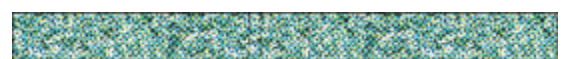
for heatless and dry grinding of ceramics and metal alloys

Silicon carbide abrasive wheels (SiC)

Magnesite bond
Unmounted
Ø of centre hole 1,8 mm
Thickness 2,0/3,0/4,5 mm
Grit size medium
Bonding hardness soft

Dry grinding, no disinfection, no sterilization; dry storage

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for medium hard metal alloys like chrome-cobalt

High-grade corundum abrasive wheels (Al₂O₃), pink

Ceramic bond
Unmounted
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness medium

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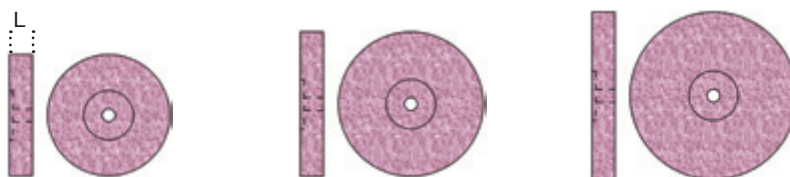
for chrome-cobalt and other metal alloys

High-grade corundum Cutting-off discs

(Al₂O₃), brown
Resin bond
Unmounted
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness hard

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for soft metal alloys

High-grade corundum abrasives (Al₂O₃), white

Ceramic bond
Grit size medium
Bonding hardness hard
Low to medium speed

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Edelkorund Schleifräder (Al₂O₃), weiß

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High-grade cor. abrasive wheels (Al₂O₃), white

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for semi-precious and non-precious metals

High-grade corundum abrasives (Al₂O₃), blue

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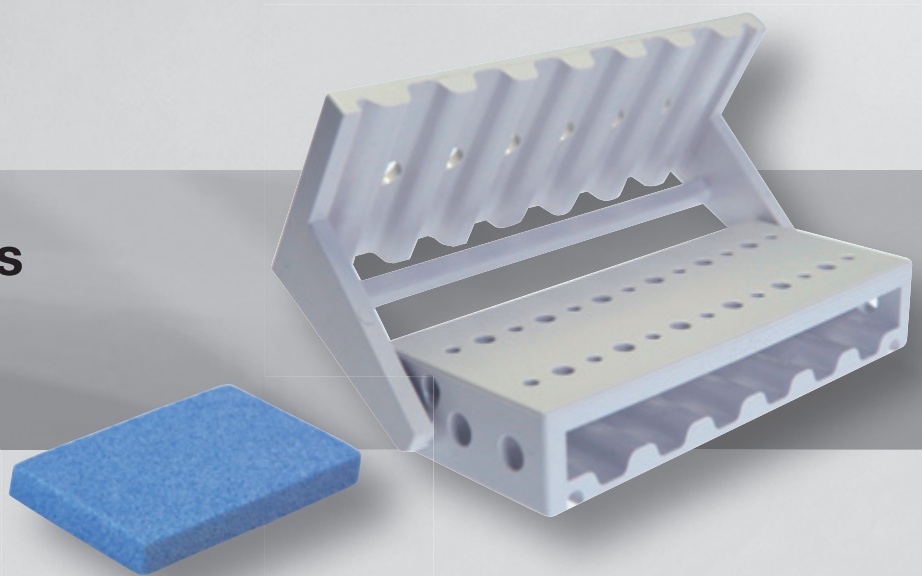
The STERI-SAFE bur block of medical resin with outstanding advantages in respect of use, safety and hygiene:

- open design for better flow penetration ensuring optimum and effective reconditioning
- the medical resin material can be disinfected and sterilized with all reconditioning measures of practical relevance
- retaining clip can be precisely swivelled and locked in place for easy handling

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See the safety, usage and hygiene
recommendations page 45

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Designed for max. 24 instruments with a total length of 36 mm to 56,5 mm

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Special advantages:

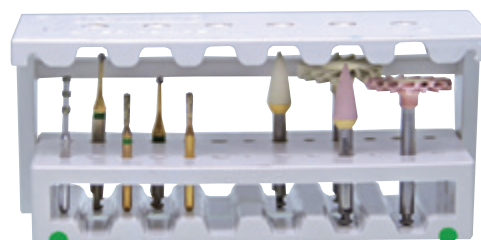
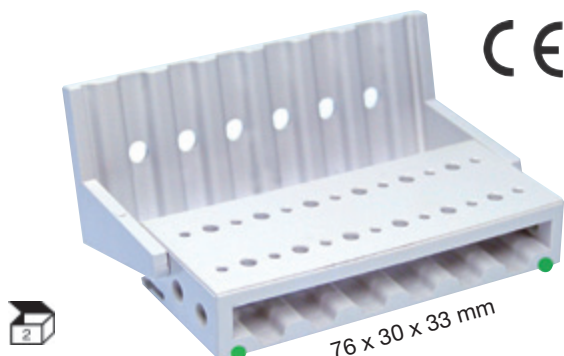
- The cleaning ability - including pre-contaminated medical products - had been validated and documented by an external institute.
- combined capacity for accommodating 14 instruments with FG shank with an overall length of 19 – 24 mm and/or 12 instruments with RA shank with an overall length of 22 – 30 mm
- can be colour-coded with silicone strips which additionally enhance the bur block's stability
- all instruments of the listed total lengths are protected against falling out when closed

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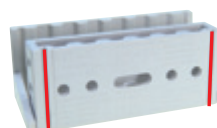
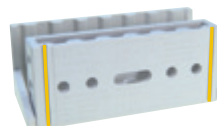
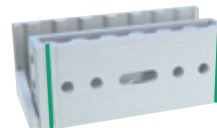
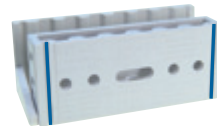
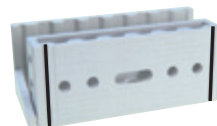
Matériau: Polystyrène,

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Assortiments possibles:

Socle double 5762:
29 orifices PM/CA + 12 orifices FG

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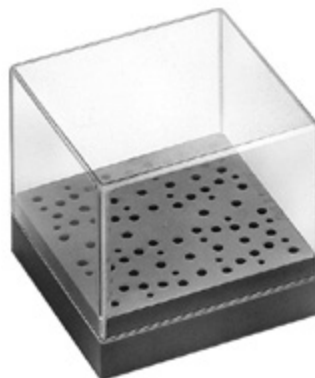
High-lustre for metals, ceramics and composites (Economical application)

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for composites, acrylics, metal alloys, chrome-cobalt and titanium

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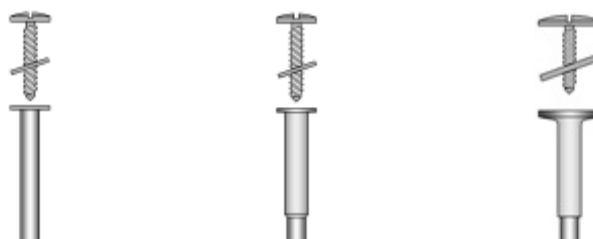
Ceramic block

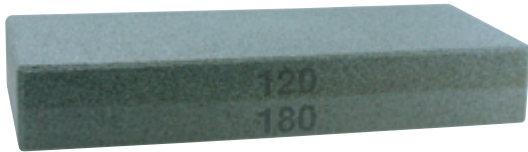
Ceramic block large, green
fine and medium grit
for dressing of polishers and
abrasives

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Ceramic block

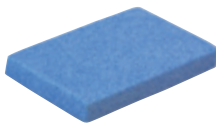
Ceramic block small, light blue
coarse grit
for dressing of polishers
and abrasives

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EVERCLEAR Mouth mirrors

Stay clear longer if shortly
warmed
Maximum reflection
Disinfectable, sterilizable

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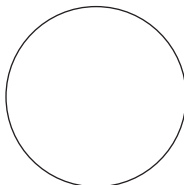
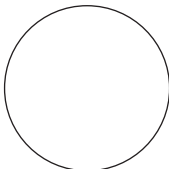
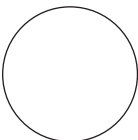
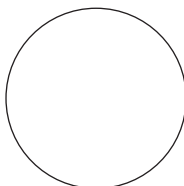
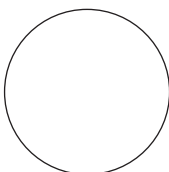
We provide each of our packagings with a detailed product label.

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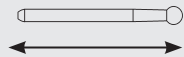
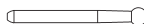
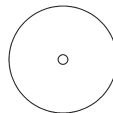
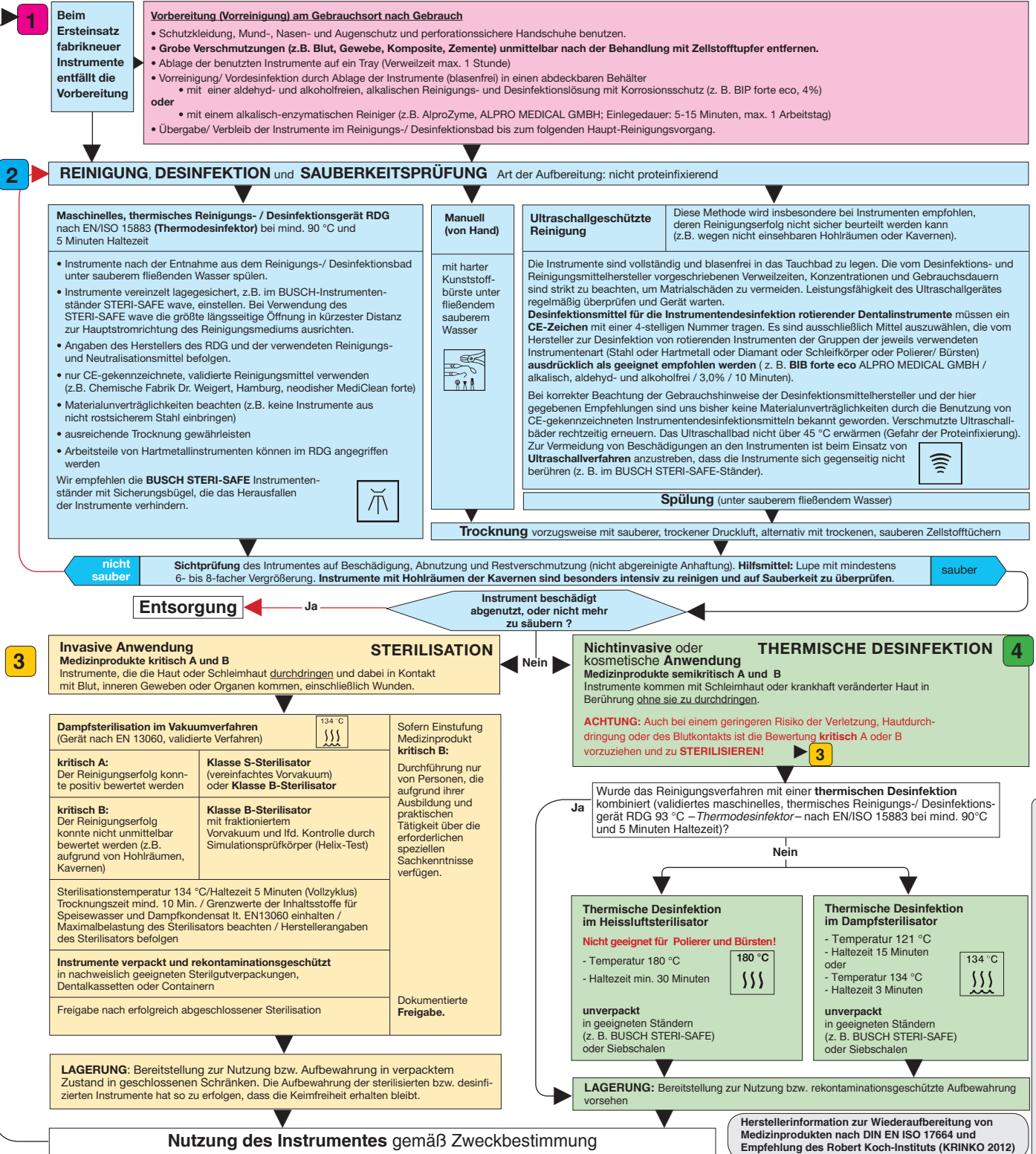
Safety recommendations

- Until they are first used, rotating instruments should be kept in their original packaging, and protected from dust and moisture, at room temperature.
- Always keep the packaging (also during the active usage period) so that the instruments are traceable if required.
- Always use fully functioning, correctly serviced and maintained as well as cleaned turbine drives as well as hand-held and angle-piece drives.
- Insert the instruments as deeply as possible.
- Before starting to work on the workpiece, bring the instruments up to operating speed and make sure that they are running concentrically.
- Observe the maximum speed (as indicated on all BUSCH packaging).
- Make sure that the instruments don't get jammed or levered.
- Depending on the application, use respiratory and/or eye protection and an extraction system.
- Do not exceed a contact pressure of 0.3 to 2.0 N.
- Only use the instruments in accordance with their intended use.
- If possible, use the entire length of the working part in order to avoid point overheating, e.g. of the tips (heat development).
- Make sure that there is sufficient water cooling to avoid unwanted heat development. For the dental practice, this means a minimum cooling flow of 50 ml per minute.
- FG instruments with a total length of more than 22 mm or a head diameter more than 2 mm might require additional cooling.
- Surgical instruments with a long shaft might require additional cooling.
- Bent or non-concentrically running instruments or instruments with damaged or worn working parts must be rejected and disposed of in order to avoid injury.

Hygiene recommendations

1 Preparation 2 Cleaning 3 Sterilisation 4 Thermal disinfection

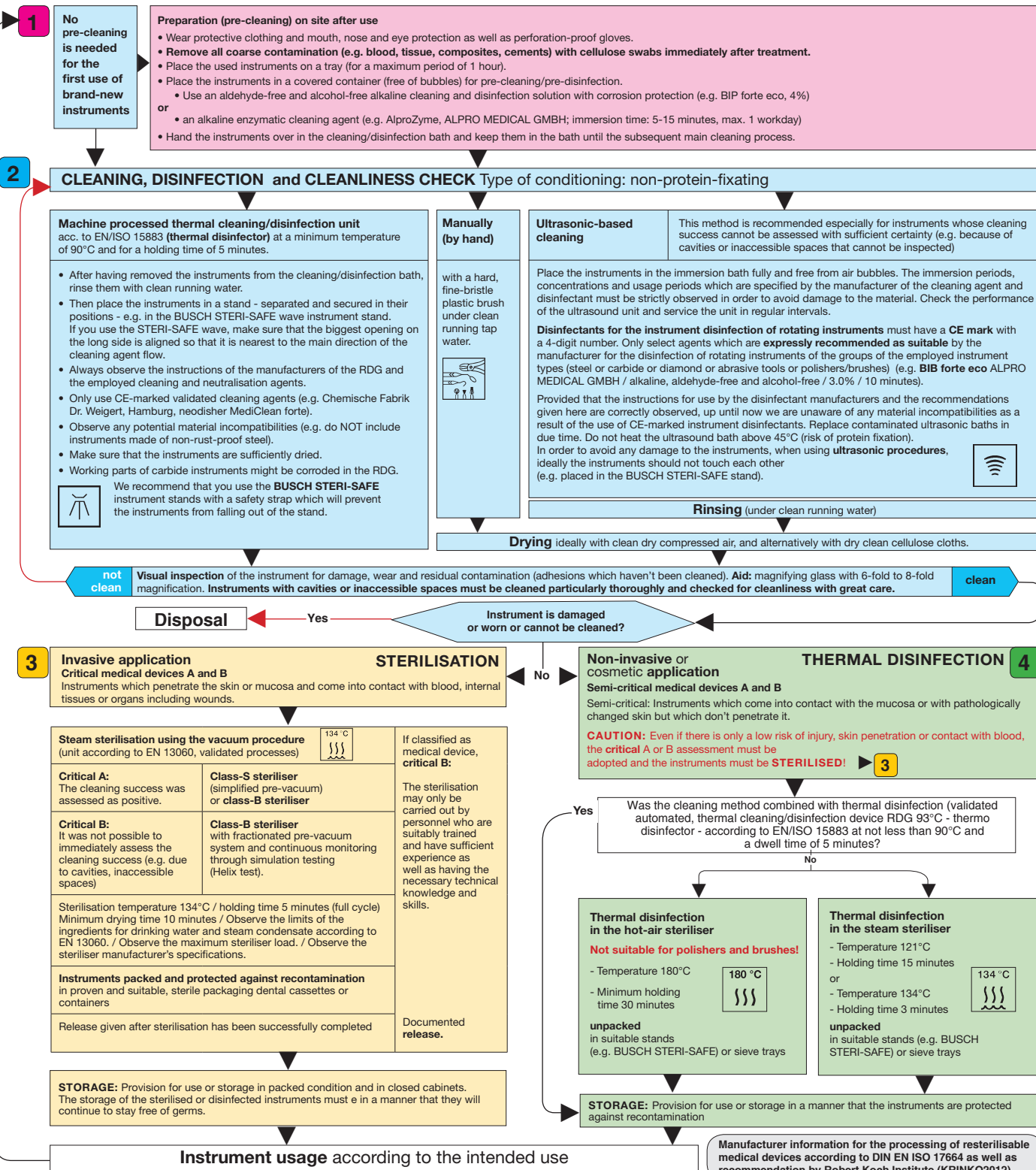
Area of application: Rotating instruments made of steel, carbide or diamond as well as polishers, abrasive tools and brushes that are intended for the application on humans.

The instruments are delivered non-sterile. Before they are used for the first time as well as after each use, they must be disinfected or cleaned and disinfected or, if required, sterilised.

Grinding caps and cap mandrels have to be disinfected and sterilized unmounted for hygienic and technical reasons.

Restrictions for the processing of resterilisable instruments: Brushes for prophylaxis are single-use products because it cannot be guaranteed that the brushes can be cleaned entirely free from residue.

For non-rust-free instruments, disinfectants and cleaning agents with corrosion protection must be used. **Non-rust-free instruments are not suitable for steam sterilisation.** The product service life is determined by wear and damage caused by the instruments' use. Therefore, specific details regarding the number of processing runs cannot be provided. Never use hydrogen peroxide (H_2O_2) for disinfection of the instruments because the risk of damage to the material cannot be excluded.

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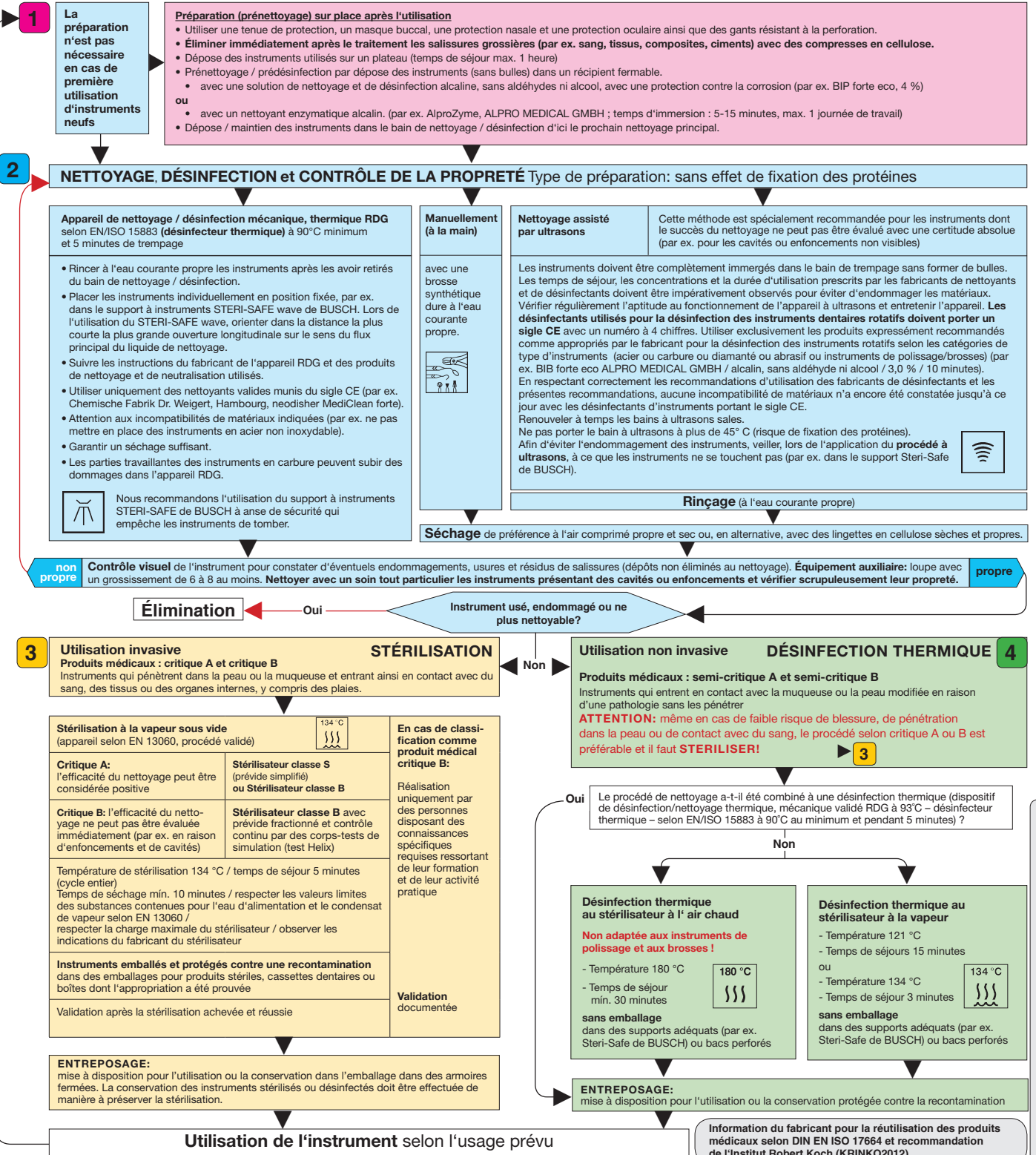
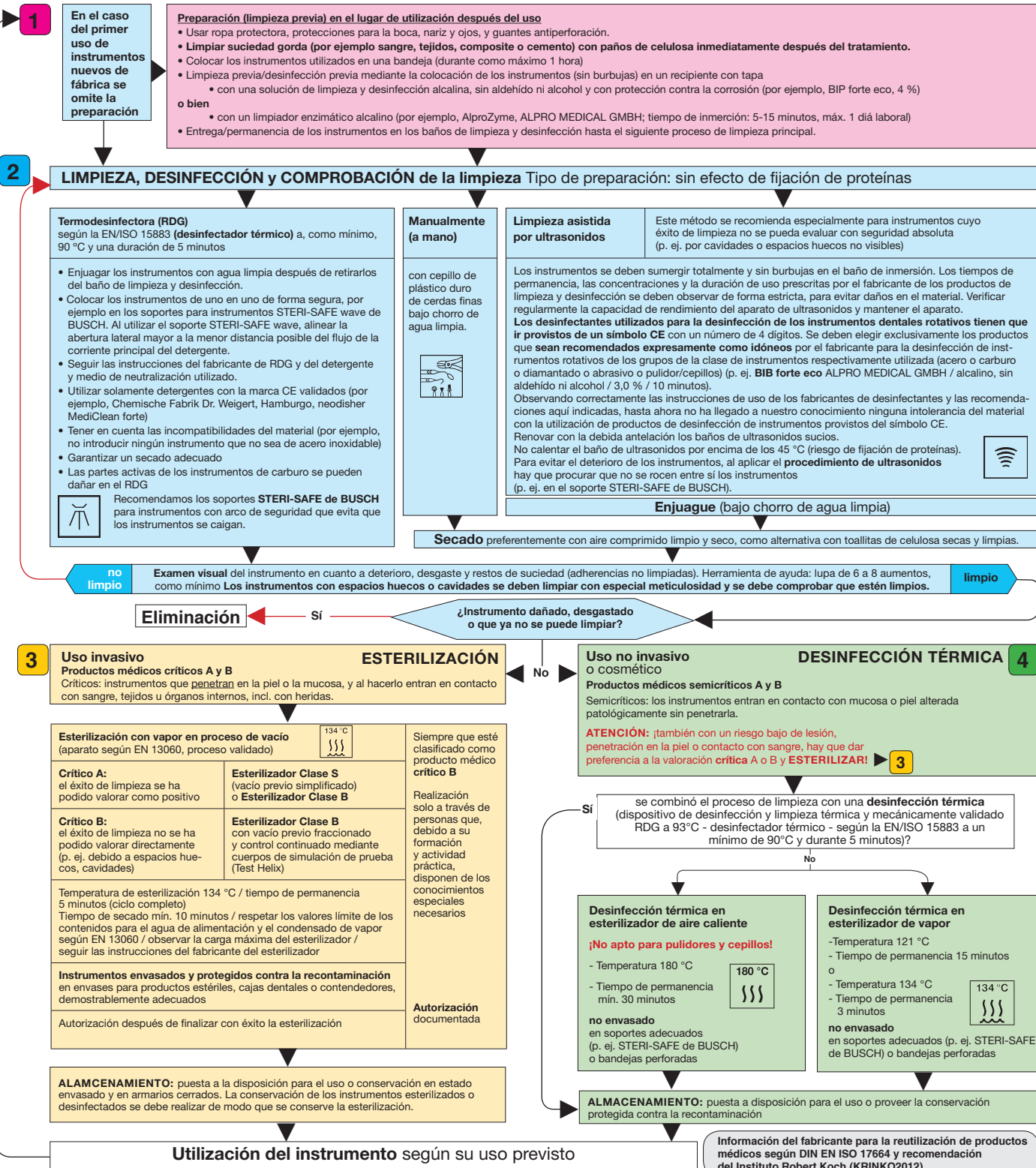
Technical data polishers, Brushes, Polishing buffs and Mandrels

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DIN, EN, ISO

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Material Polishers

Silicone, Polyurethan (PUR), Polyester

Material Brushes

Dentistry: Polyamide
Dental Technique: Bristle

Polishing buffs

Dental Technique
cambric, chamois, cotton
Microfiber

Mandrels

high-grade special steel,
stainless or nickel plated

Bond hardness

super soft, soft,
medium, hard

Abrading medium

Silicone carbide (SiC)
Diamond
High grade corundum
Pumice

Grit size

extra fine, fine, medium,
coarse

Shank material

stainless steel or
nickel-plated
Plastic

Total length, Shank ISO

HP 47,0 - 58,0 mm 104
RA 22,0 - 31,0 mm 204
FG 21,0 - 25,0 mm 314

Shank diameter

HP 2,35 mm - 0,016 mm
RA 2,35 mm - 0,016 mm
FG 1,60 mm - 0,010 mm

Working part diameter

3,0 - 22,0 mm
The ISO diameter is
mentioned in one 10th
of a millimeter.

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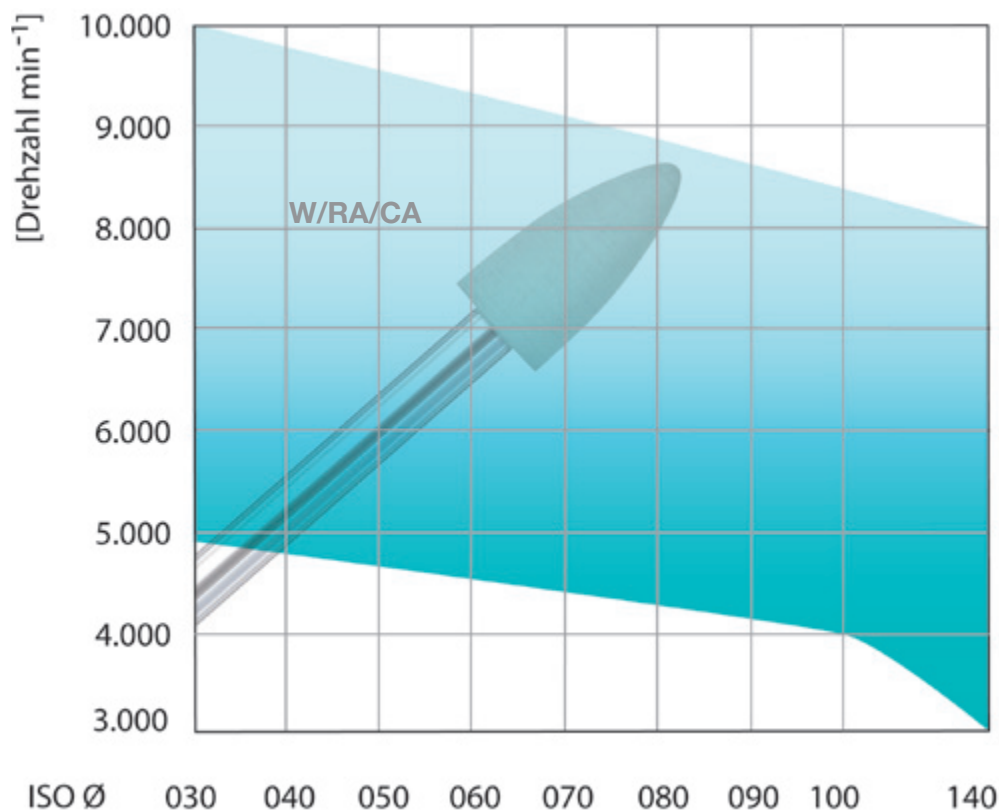
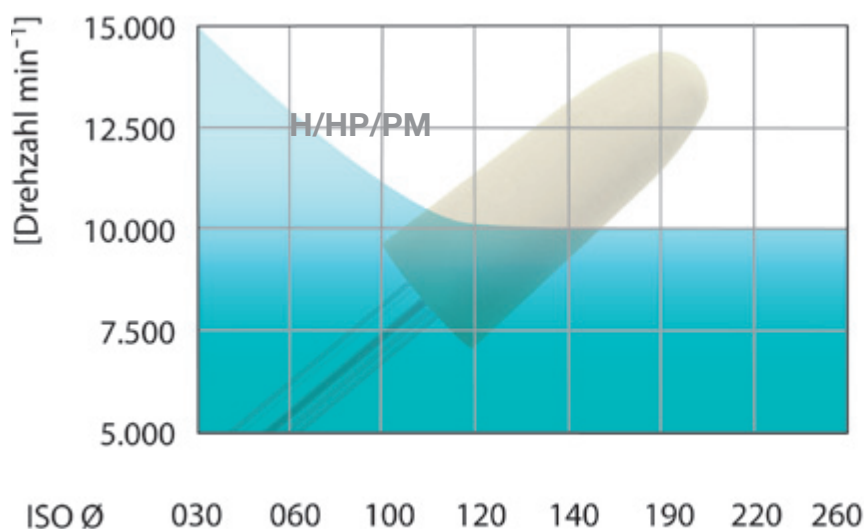
(FG Ø 030: 3.000–8.000 min⁻¹) with water spray. If not sufficient cooling medium is used, irreversible damages of the teeth and the surrounding tissue can happen.

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Siliziumkarbid (SiC)
Edelkorund (Al₂O₃)
Diamant

Abrasive

Silicon carbide (SiC)
High-grade corundum (Al₂O₃)
Diamond

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Bonding

Ceramic, magnesite, resin
Resin
synthetic

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PM 2,35 - 0,016 mm
CA 2,35 - 0,016 mm
FG 1,60 - 0,010 mm

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Working part diameter

1,8 - 75,0 mm
The ISO diameter is
mentioned in one 10th
of a millimeter.

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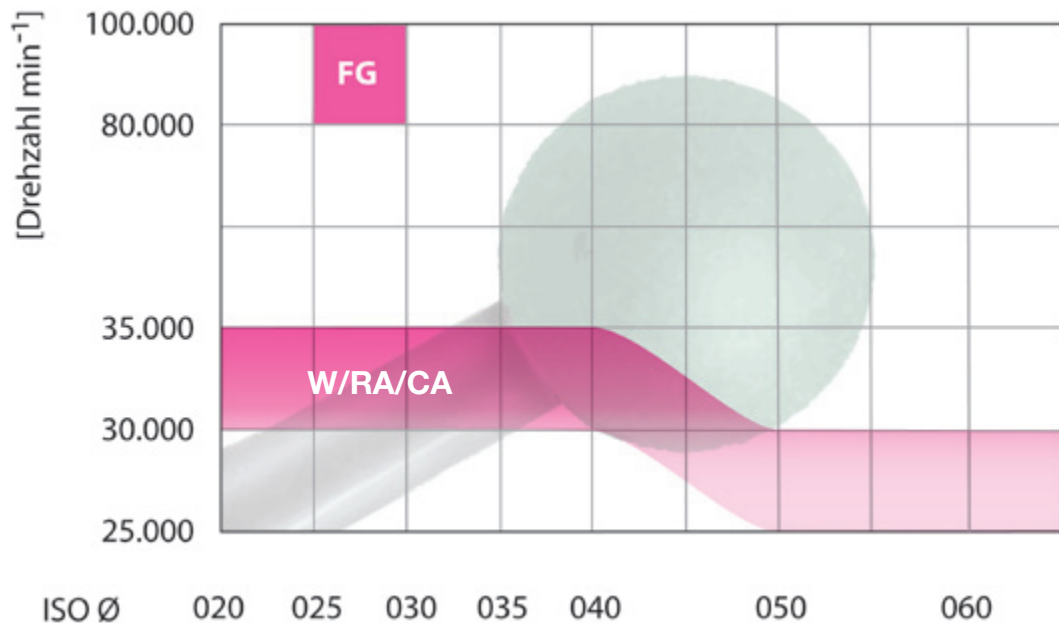
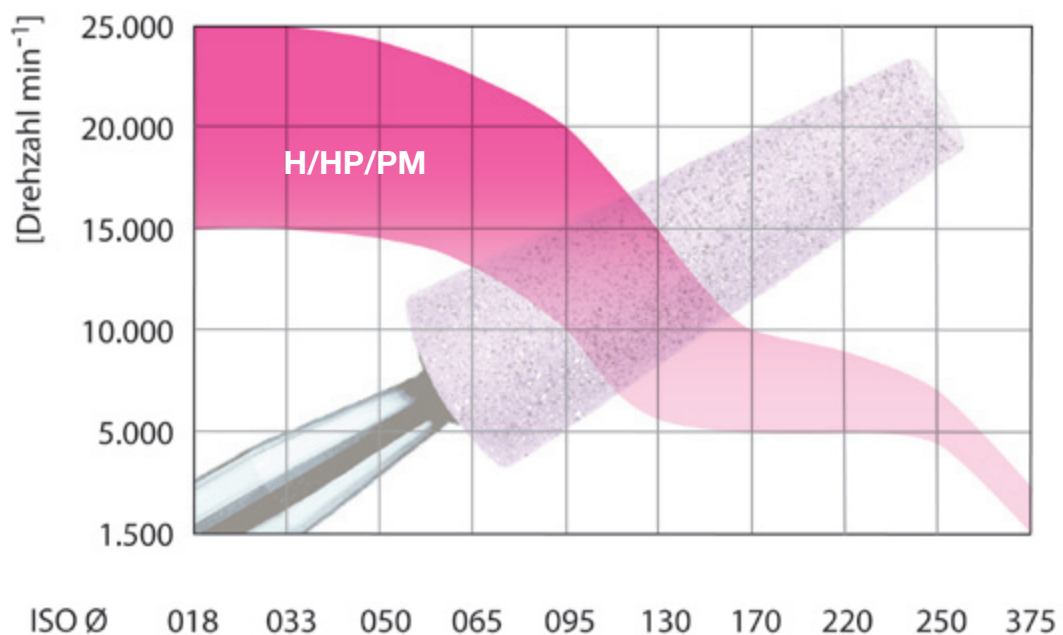
Instrumentensätze, Sonstiges

Sets of instruments, miscellaneous

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Illustrations and photos are protected by copyright. Reproductions, also by extract in any form, are only permitted with written authorization of BUSCH & CO. GmbH & Co. KG Engelskirchen.

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To reserve the right to modify our range of products and their design in the sense of progress are traditions of our company.

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Quality management

- ISO 13485:2012+AC:2012
- MDD93/42/EEC A-II/excl.(4)

VDDI
Dental Solutions.
German Manufacturers.