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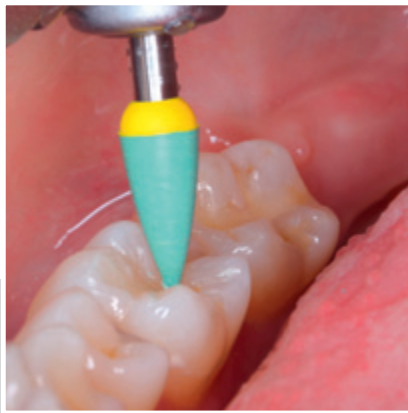
Dentistry / Dental Technique
Odontologie / Prothese Dentaire
Odontologia / Protesis Dental

Polierer und Schleifkörper
Polishers and abrasives
Polissoirs et abrasifs
Pulidores y abrasivos

2015/2016

There is no substitute for quality

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Our company's
philosophy is our personal
standard.

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Universal pre-polishers, for all restoration materials

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2-step diamond interspersed polishing system for composite (also high filled)

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Polishing, pink beige

Diamond, PUR
Grit size medium
Bond hardness medium

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2-step polishing system for composite

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Polishing, yellow

High-grade corundum Silic. bond
Grit size medium
Bond hardness soft

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Also suitable for the removal of
adhesive residues after bracket
treatment.

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End-polishing, blue

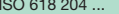
High-grade corundum,
Silicone bond
Grit size fine
Bond hardness soft

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2-step polishing system for precious and non-precious metal alloys, amalgam

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Polishing, brown

SiC, PUR
Grit size medium
Bond hardness medium

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End-polishing, green

SiC, PUR
Grit size fine
Bond hardness soft

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2-step diamond interspersed polishing system for ceramics

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End-polishing, green yellow

Diamant, PUR
Grit size fine
Bond hardness soft

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2-step diamond interspersed polishing system for zirconium oxide

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Polishers and brushes for prophylaxis

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for the application with polishing paste, white

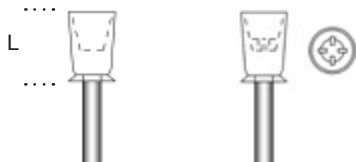
Silicone polisher
(without abrasive grain)
Bond hardness medium

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BIMS PUR,
Korngröße extra fein

for the application without polishing paste, white, green, yellow

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mittel
medium
moyen
medio

soft
soft
soft
blando

super soft
super soft
super soft
super-blando

W/RA/CA ISO 003 204 ...	9633M/070 034 503 070	9633S/070 034 502 070	9633SS/070 034 501 070
L mm	10,00	10,00	10,00

Einmalinstrument / Kunststoffschaft
Single use instrument / plastic shank
Usage unique /
Tige en matière plastique
Uzar una vez / material del mango de plástico

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for the application with polishing paste, white

Polyamide brushes
(without abrasive grit)

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for the application without polishing paste, beige

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Siliziumkarbid

Schleifkörper (SiC), grün

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for ceramic, acrylics, metal alloys, enamel

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Edelkorund Schleifkörper (Al₂O₃), rosa

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for medium hard metal alloys

High-grade corundum abrasives (Al₂O₃), pink

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**Edelkorund Arkansas
Schleifkörper (Al₂O₃),
weiß**

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for smooth grinding of composites

**High-grade corundum
Arkansas abrasives (Al₂O₃),
white**

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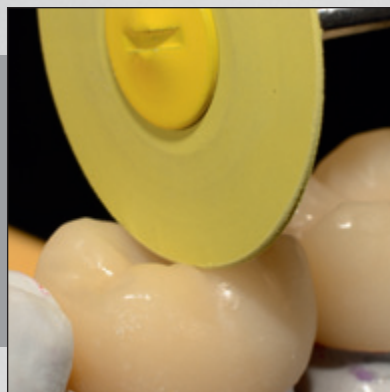
Universal pre-polishers for all materials

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Smoothing, white

SiC, Silicone
Grit size coarse
Bond hardness soft

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Pre-polishing for model casting, CrCo, mocha

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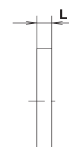
2-step polishing system for precious metal alloys

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SOFT WHEELS; impregnated buffs for metals

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Impregnated cotton buffs (application without polishing paste)

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Cambric, fine nettle cloth
for high-lustre polishing
with diamond paste.

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Chamois, for high-lustre
polishing after the use of
paste.

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Cotton, for the final high-
lustre polishing after the use
of paste.

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Microfiber for the final high-
lustre polishing after using
paste.

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for ceramics

Silicon carbide abrasives (SiC), green

Ceramic bond
Grit size fine
Bonding hardness medium

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for ceramics, acrylics, metal alloys

Silicon carbide abrasives (SiC), green

Ceramic bond
Unmounted,
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness medium

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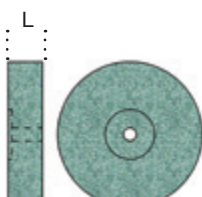
for precious metal alloys and ceramics

**Silicon carbide
abrasives (SiC), green**

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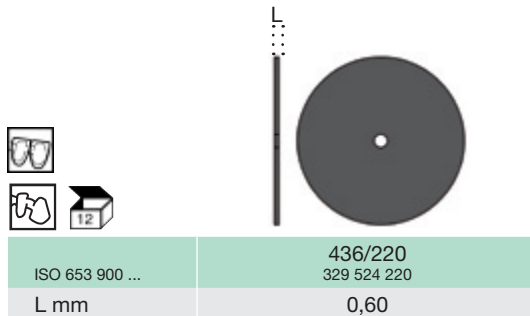
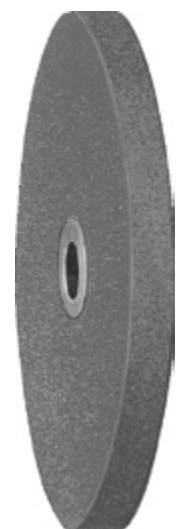
for acrylics and metal alloys

**Silicon carbide
abrasives (SiC),
green**

Ceramic bond
Unmounted
Ø of centre hole 6,5 mm
Thickness 6,5 mm
Grit size medium, coarse
Bonding hardness
medium

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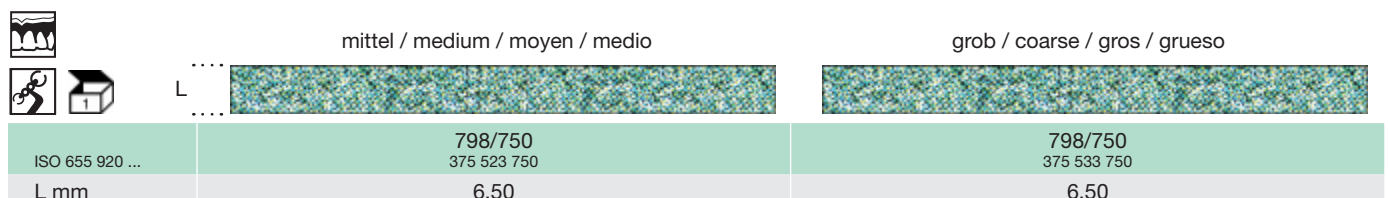
for heatless and dry grinding of ceramics and metal alloys

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for finest grinding of ceramics and metal alloys

Silicon carbide abrasive wheels (SiC)

Magnesite bond
Unmounted
Ø of centre hole 1,8 mm
Thickness 2,0/3,0/4,5 mm
Grit size fine
Bonding hardness soft

**Dry grinding,
no disinfection,
no sterilization;
dry storage**

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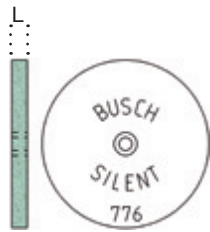
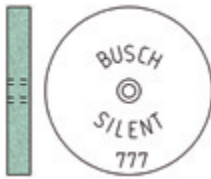
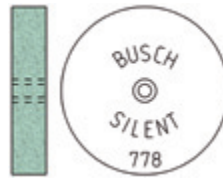
Edelkorund Schleifkörper (Al_2O_3), rosa

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for medium hard metal alloys

High-grade corundum abrasives (Al_2O_3), pink

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for medium hard metal alloys like chrome-cobalt

High-grade corundum abrasives (Al₂O₃), pink
Ceramic bond
Unmounted
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness medium

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for hard metal alloys and chrome-cobalt

High-grade corundum abrasives (Al₂O₃), brown

Ceramic bond
Grit size medium
Bonding hardness soft
Medium to high speed

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for chrome-cobalt and other metal alloys

High-grade corundum abrasives (Al₂O₃), brown

Resin bond
Unmounted
Ø of centre hole 1,8 mm
Grit size medium
Bonding hardness hard

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for soft metal alloys

High-grade corundum abrasives (Al₂O₃), white

Ceramic bond
Grit size medium
Bonding hardness hard
Low to medium speed

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Edelkorund Schleifräder (Al₂O₃), weiß

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High-grade cor. abrasive wheels (Al₂O₃), white

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for semi-precious and non-precious metals

High-grade corundum abrasives (Al₂O₃), blue

Ceramic bond
Grit size medium
Bonding hardness medium

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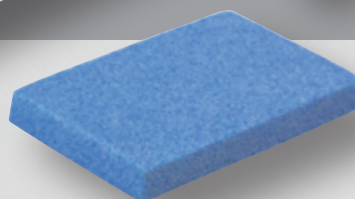
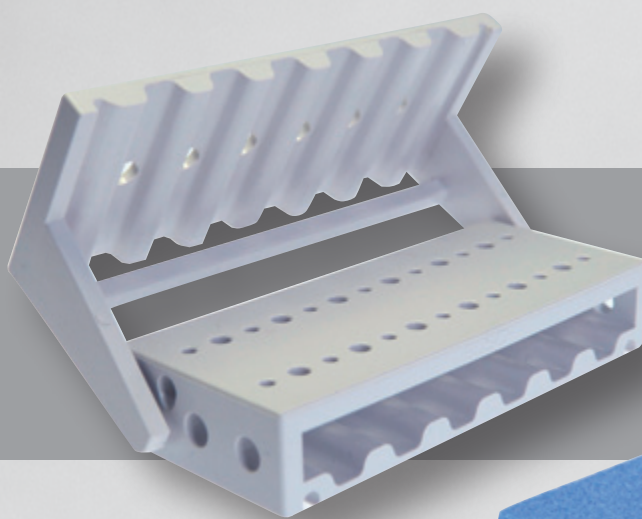
The STERI-SAFE bur block of medical resin with outstanding advantages in respect of use, safety and hygiene:

- open design for better flow penetration ensuring optimum and effective reconditioning
- the medical resin material can be disinfected and sterilized with all reconditioning measures of practical relevance
- retaining clip can be precisely swivelled and locked in place for easy handling

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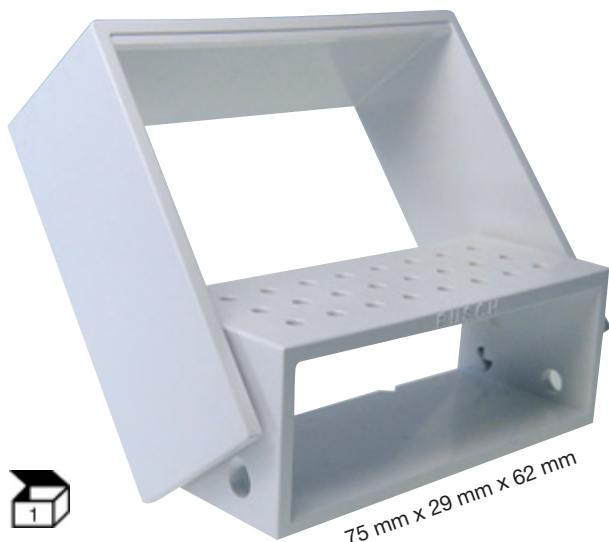
Designed for max. 24 instruments with a total length of 36 mm to 56,5 mm

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Special advantages:

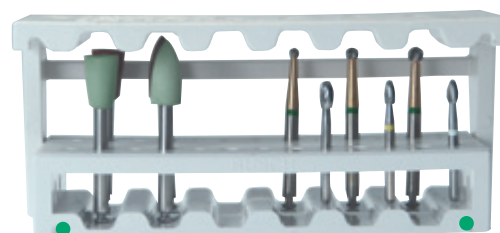
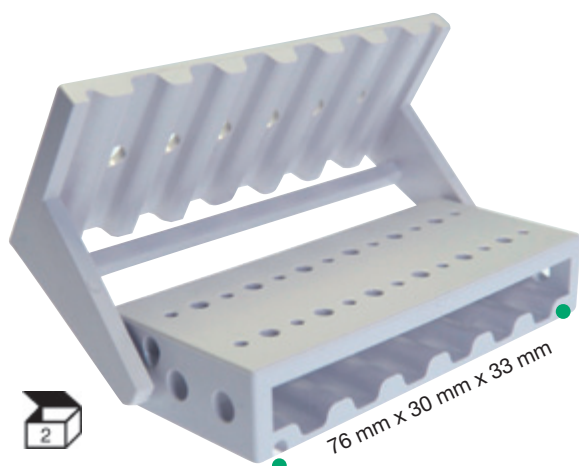
- combined capacity for accommodating 14 instruments with FG shank with an overall length of 19 – 24 mm and/or 12 instruments with RA shank with an overall length of 22 – 30 mm
- can be colour-coded with silicone strips which additionally enhance the bur block's stability
- all instruments of the listed total lengths are protected against falling out when closed

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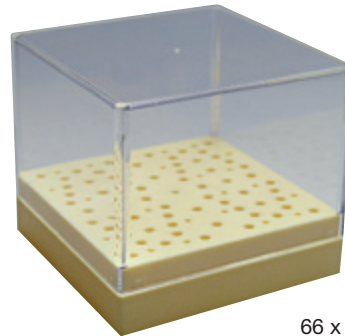
High-lustre for metals, ceramics and composites (Economical application)

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for composites, acrylics, metal alloys, chrome-cobalt and titanium

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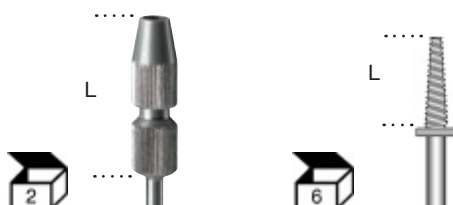
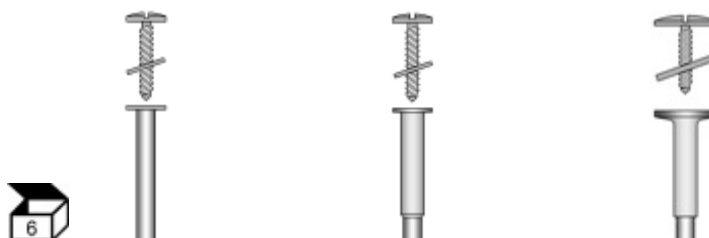
Ceramic block

Ceramic block large, green
fine and medium grit
for dressing of polishers and
abrasives

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Ceramic block

Ceramic block small, light blue
coarse grit
for dressing of polishers and
abrasives

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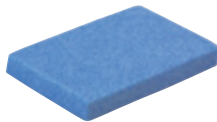
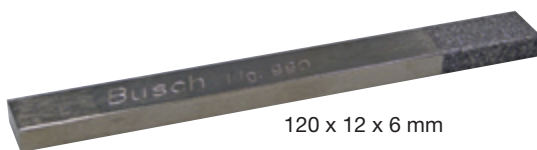
EVERCLEAR Mouth mirrors

Stay clear longer if shortly
warmed
Maximum reflection
Disinfectable, sterilizable

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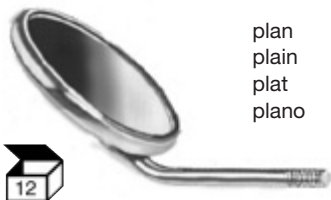
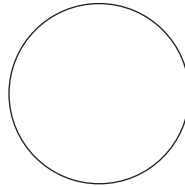
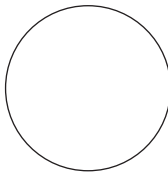
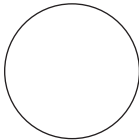
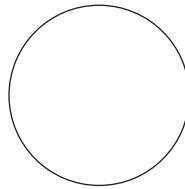
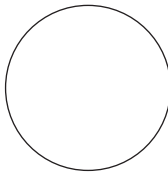
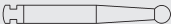
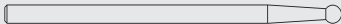
Silicon carbide abrasives
(SiC), green
Ceramic bond
Grit size medium,
Bonding hardness medium

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We provide each of our packagings with a detailed product label.

On this label is noted:
a.) - c.) HIBC-Code
d.) + e.) details about the BUSCH-partner
f.) CE-mark for medical products
g.) advice on further information in BUSCH-catalogues; home-page
h.) LOT-number
i.) max. admissible rpm
j.) material-and shank description
k.) Busch-number and size.
For your information and safety.

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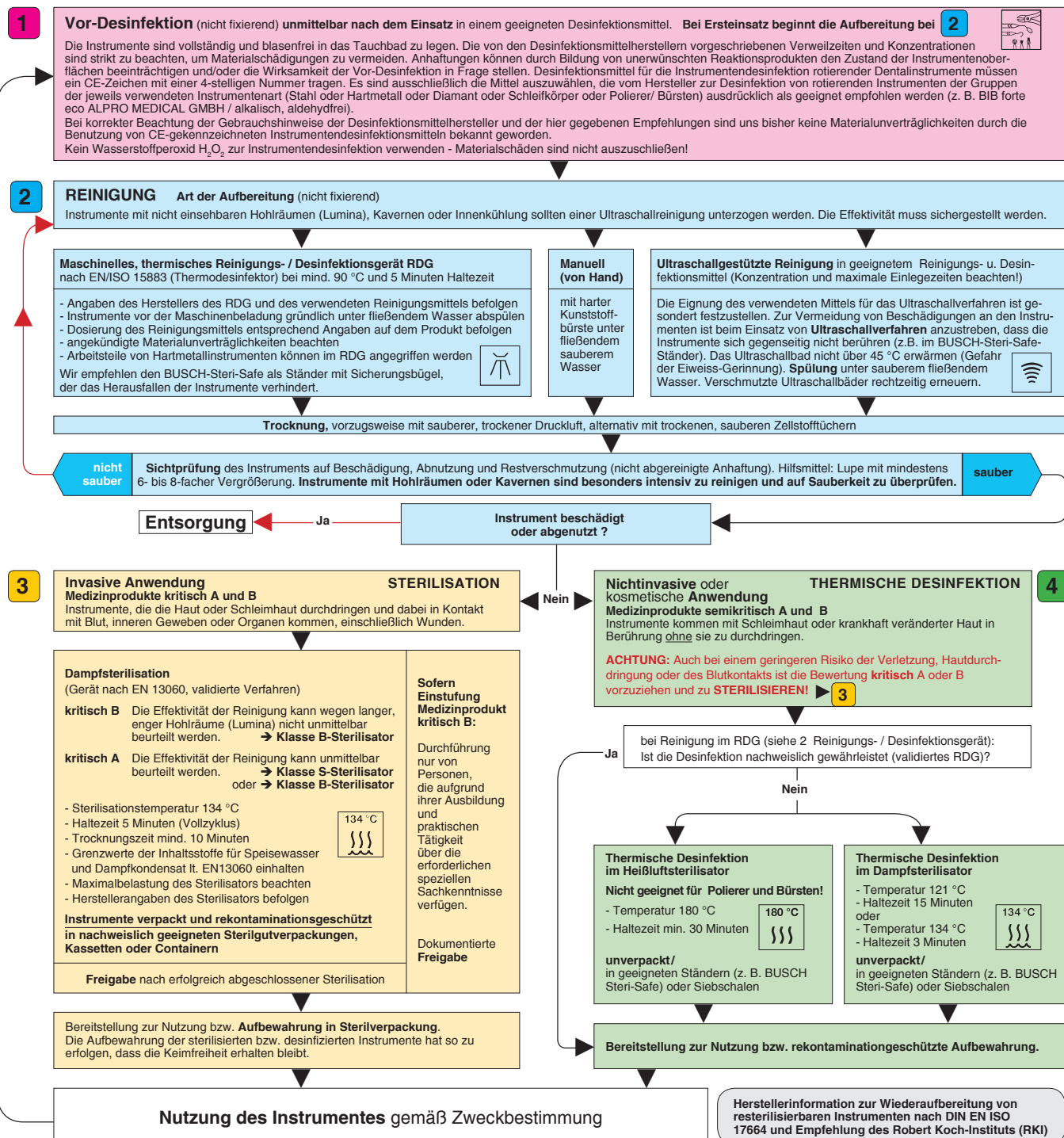
Safety recommendations

- Rotary instruments should be stored at room temperature in their original packaging where they are protected from dust and moisture until used for the first time.
- The packaging should be retained even while the instruments are being used so that they can be traced if necessary.
- Only use undamaged turbines which have been serviced and cleaned.
- Instruments must be inserted as deep as possible.
- Instruments must have reached the required speed before being applied to the workpiece; absolutely concentric running must be ensured.
- Note the maximum permitted rotating speed (specified on every BUSCH package).
- Do not wedge or lever the instruments.
- Wear a mask / safety glasses and use aspiration, depending on the application concerned.
- Do not exceed the maximum contact pressure of 0.3 to 2 N.
- Instruments may only be used in conformity with their intended use.
- Use the full working part length where possible in order to avoid local overloading of the tips, for example (build-up of heat).
- Adequate cooling with water (at least 50 ml per minute in dental practice) must be ensured to prevent an undesirable build-up of heat.
- Additional cooling is required for FG instruments with a total length of more than 22 mm or a working part diameter greater than 2 mm.
- Additional cooling may be required for surgical instruments with long shank.
- Deformed or non-concentric instruments and instruments with damaged / worn working parts must be discarded and replaced.

Hygiene recommendations

1 Pre-disinfection

2 Cleaning

3 Sterilization

4 Thermal disinfection

Application area: Rotary steel, carbide and diamond instruments, polishers, abrasives and brushes for use on patients. The instruments are delivered in non-sterile form. They must be disinfected, cleaned and if necessary sterilized before being used for the first time and after every use.
Restricted re-conditioning: Prophylaxis brushes are not reusable as total cleaning cannot be guaranteed. Disinfectants and cleaning agents with corrosion protection must be used for non-stainless instruments. Non-stainless instruments cannot be autoclaved. Product life depends on wear and damage due to use of the instruments – the number of conditioning cycles therefore cannot be stated in absolute terms. medCap and medCap carrier must always be re-conditioned separately.

1

Pre-disinfection (non-fixing) immediately after use, using a suitable disinfectant. For new instruments which are to be used for the first time, start with 2

The instruments must be immersed in the bath completely and bubble-free. The dwell times and concentrations specified by the disinfectant manufacturers must be scrupulously observed in order to avoid damaging the material. Adhering deposits which have not been removed may form undesirable reaction products impairing the surface condition of the instruments and/or make pre-disinfection ineffective. Disinfectants for rotary dental instruments must bear a CE mark with a four-digit number. Only products which have explicitly been recommended by their manufacturer for disinfecting rotary instruments of the respective types (steel or carbide or diamond or abrasives or polishers / brushes) may be used for this purpose (e.g. BiB forte eco ALPRO MEDICAL GMBH / alkaline, aldehyde-free).

Material incompatibility due to the use of instrument disinfectants bearing the CE mark have so far not been reported, provided that the disinfectant manufacturer's instructions for use and the recommendations set out here are correctly observed.

Hydrogen peroxide H₂O₂ must not be used to disinfect dental instruments, as damage to the material cannot be excluded!

2

CLEANING Type of conditioning (non-fixing)

Instruments with hidden cavities (lumina), caverns or internal cooling should be cleaned in an ultrasonic process. Effective cleaning must be assured.

Mechanical, thermal washer / disinfectant in accordance with EN ISO 15883 (thermal washer/disinfectant) with at least 90 °C and 5 minutes dwell time

- Follow the instructions given by the manufacturer of the machine and cleaning agent used
- Thoroughly rinse the instruments under running water before placing them in the machine
- Follow the dosage instructions for cleaning agent specified on the product
- Note any indicated material incompatibility
- Working parts of carbide instruments may be corroded in the machine

We recommend using the BUSCH-Steri-Safe as stand with retaining clip to prevent the instruments from falling out.

Manually

with a hard plastic brush under clean running water

Ultrasonic cleaning using a suitable cleaning agent and disinfectant (consider the concentrations and maximum dwell times!)

The intended product's suitability for use in an ultrasonic process must be established separately. In order to prevent damage to the instruments when using ultrasonic methods, care should be taken to prevent the instruments from touching one another (e.g. by using the BUSCH-Steri-Safe stand). Do not heat the ultrasonic bath over 45 °C (risk of protein coagulation). Rinse under clean running water. Contaminated ultrasonic baths must be replaced in due time.

Dry, preferably with clean, dry compressed air, alternatively with soft, clean paper towels.

not clean

Visual inspection of the instrument for signs of damage, wear and remaining dirt (adhering deposits which have not been removed). Aid: magnifying glass with at least sixfold to eightfold magnification. Instruments with hollow spaces or caverns must be cleaned with particular intensity and examined to ensure they are clean.

clean

Disposal

Instrument damaged or worn out?

Yes

3

Invasive use
Critical medical devices A and B

Instruments which penetrate the skin or mucous membrane and come into contact with blood, internal tissues or organs, including wounds.

STERILIZATION

Steam sterilization

(device in accordance with EN 13060, validated procedure)

Critical B Effective cleaning cannot be assessed directly on account of long, narrow cavities (lumina).
→ Grade B sterilizer

Critical A Effective cleaning can be assessed directly.
→ Grade S sterilizer
or
→ Grade B sterilizer

- Sterilization temperature 134 °C
- Dwell time 5 minutes (full cycle)
- Drying time at least 10 minutes
- Limit values of the constituents for feed water and steam condensate specified in EN13060 must be observed
- Note maximum loading of the sterilizer
- Follow the sterilizers manufacturer's instructions

Instruments packed and protected against recontamination in demonstrably suitable packaging for sterilized products, cassettes or containers

Released for use after successful sterilization

If classified as critical medical device B:

May only be undertaken by duly trained, qualified and experienced personnel with the requisite special know-how.

Documented release

Availability for use and storage in sterile packaging.

The sterilized and disinfected instruments must be stored in such a way as to exclude all risk of recontamination

Use of the instrument in accordance with its intended use

Non-invasive or cosmetic application
Semi-critical medical devices A and B

Instruments in contact with mucous membrane or pathologically altered skin without penetrating it.

IMPORTANT: Even if the risk of injury, penetration of the skin or contact with blood is lower, a critical A or B rating should be preferred and the instruments **STERILIZED!**

→ 3

THERMAL DISINFECTION

If cleaned in the thermal washer/disinfectant:
Is disinfection demonstrably assured (validated device)?

Yes

No

Thermal disinfection in a hot-air sterilizer

Not suitable for polishers and brushes!

- Temperature 180 °C

- Min. dwell time 30 minutes

Without packaging /

in suitable stands (e.g. BUSCH Steri-Safe) or net baskets

Thermal disinfection in an autoclave

- Temperature 121 °C

- Dwell time 15 minutes

or

- Temperature 134 °C

- Dwell time 3 minutes

Without packaging /

in suitable stands (e.g. BUSCH Steri-Safe) or net baskets

Availability for use or storage protected against recontamination.

Manufacturer's information for reconditioning instruments which can be resterilized in accordance with DIN EN ISO 17664 and the recommendations of the Robert Koch-Institute (RKI) Germany

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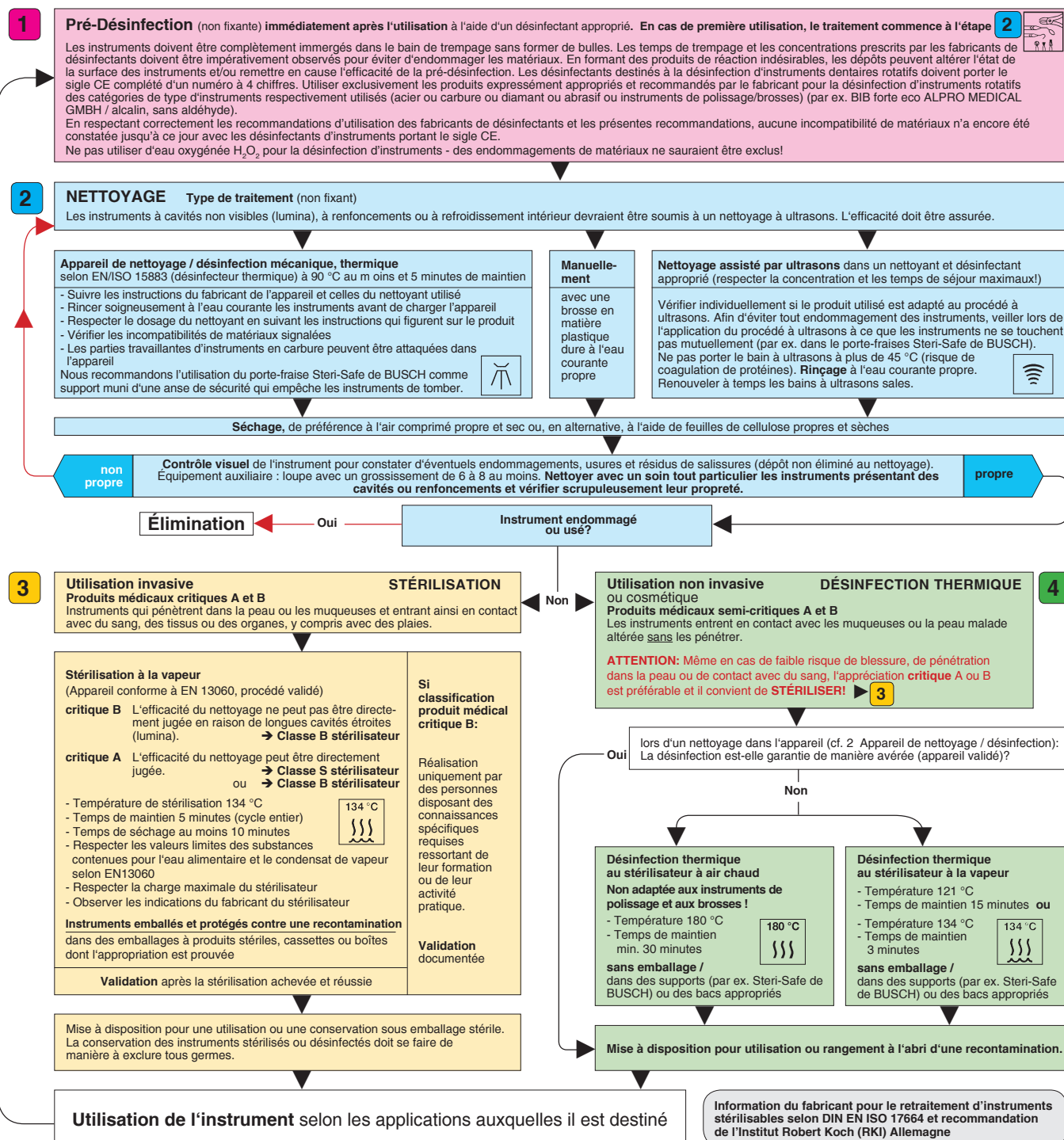
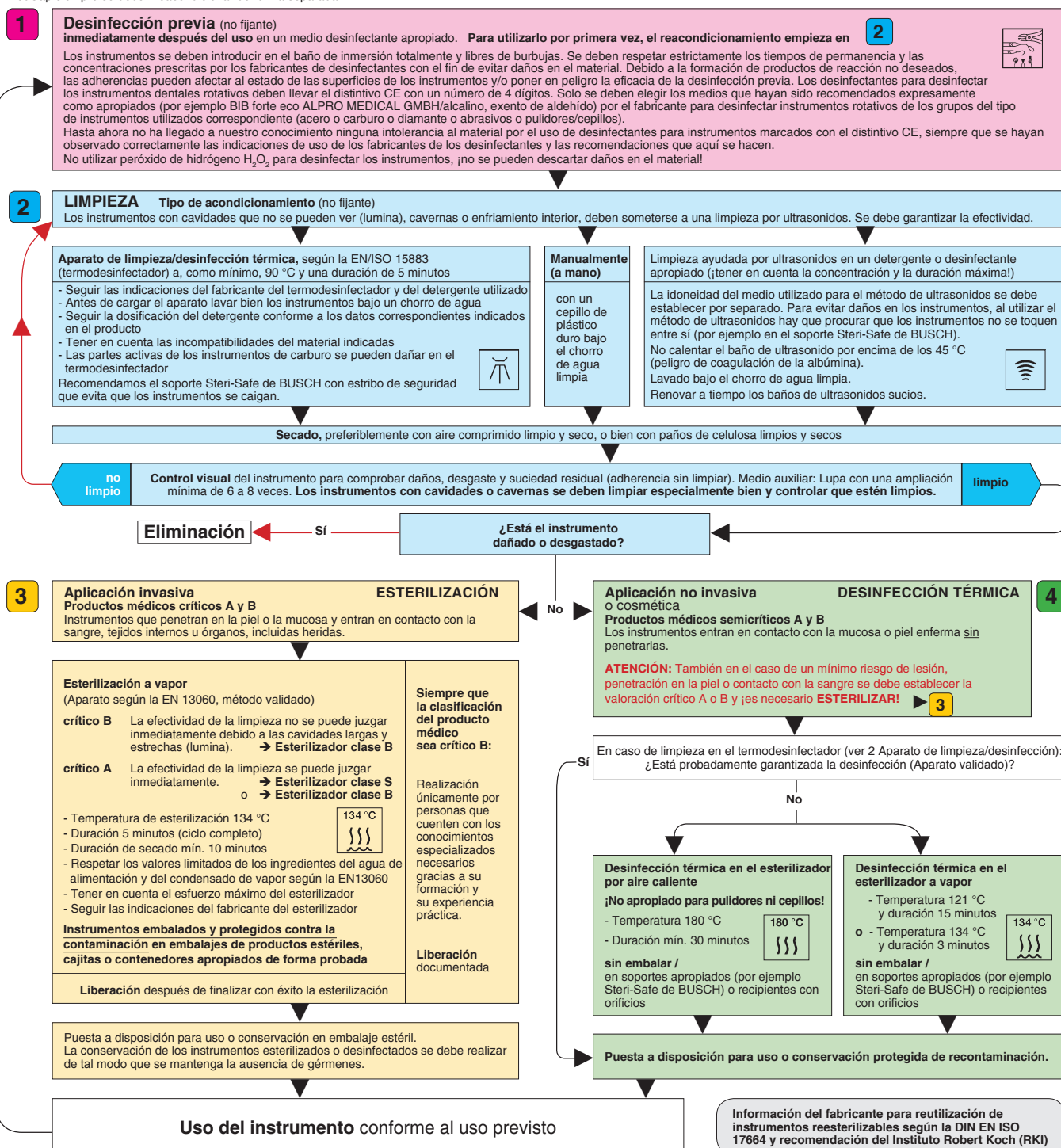
Silikon, Polyurethan (PUR), Polyester

Material Polishers

Silicone, Polyurethan (PUR), Polyester

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Material Brushes

Dentistry: Polyamide
Dental Technique: Bristle

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Mandrels

high-grade special steel, stainless or nickel plated

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Shank material

stainless steel or nickel-plated
Plastic

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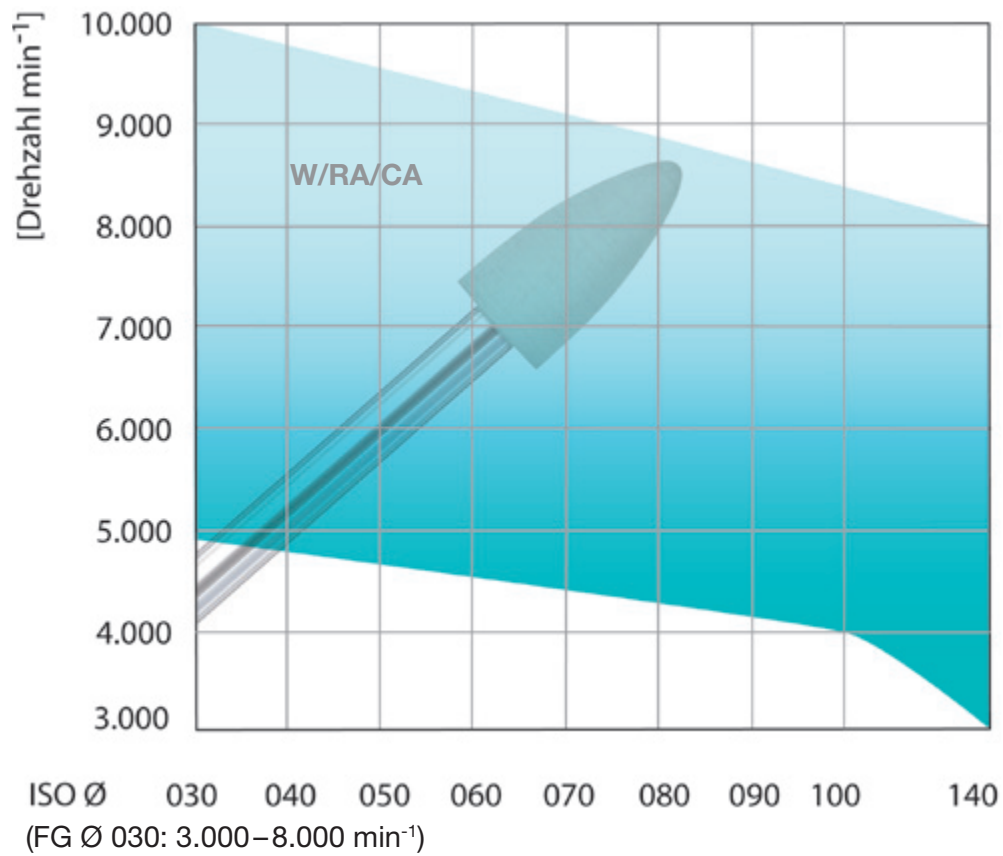
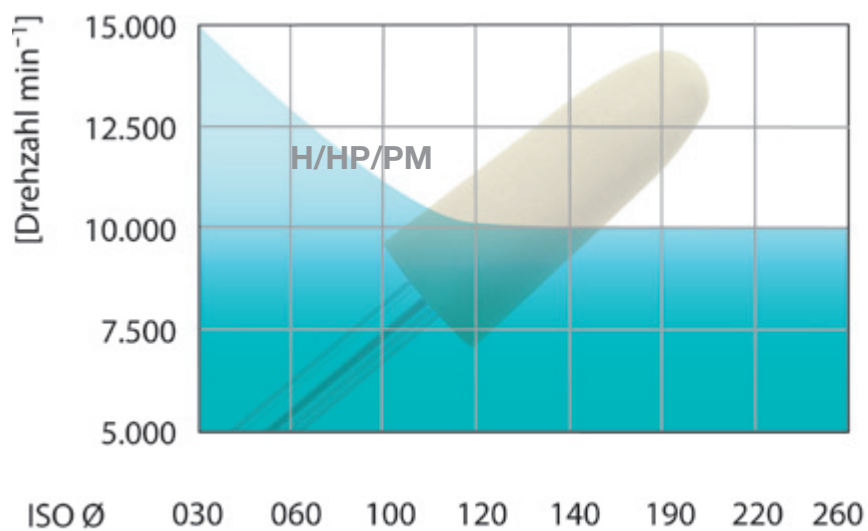
Siliziumkarbid (SiC)
Edelkorund (Al₂O₃)
Diamant

Abrasive

Silicon carbide (SiC)
High-grade corundum (Al₂O₃)
Diamond

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Bonding

Ceramic, magnesite, resin
Resin
synthetic

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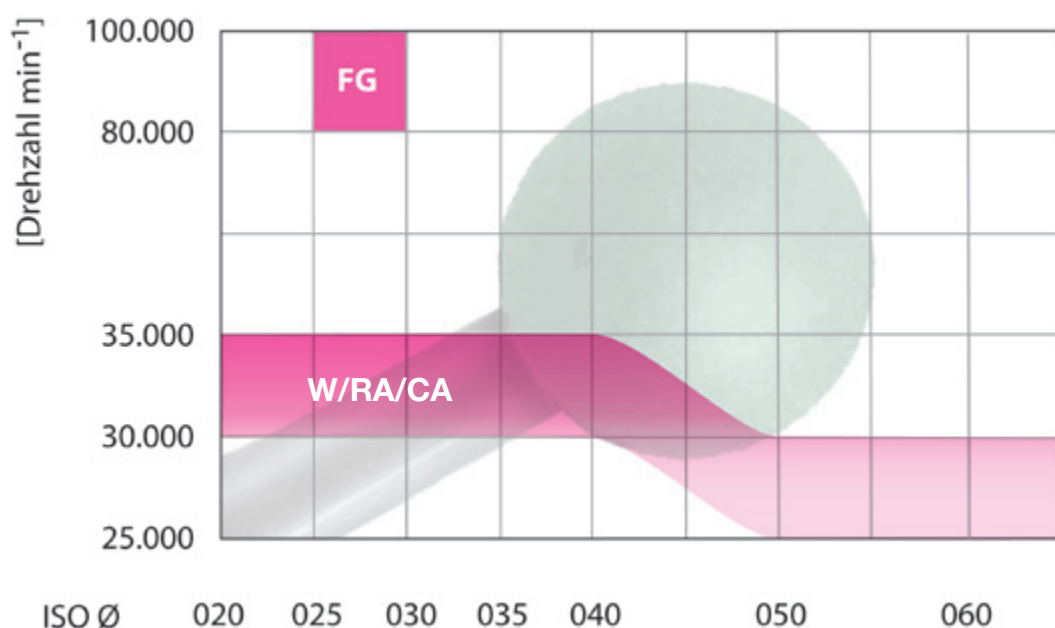
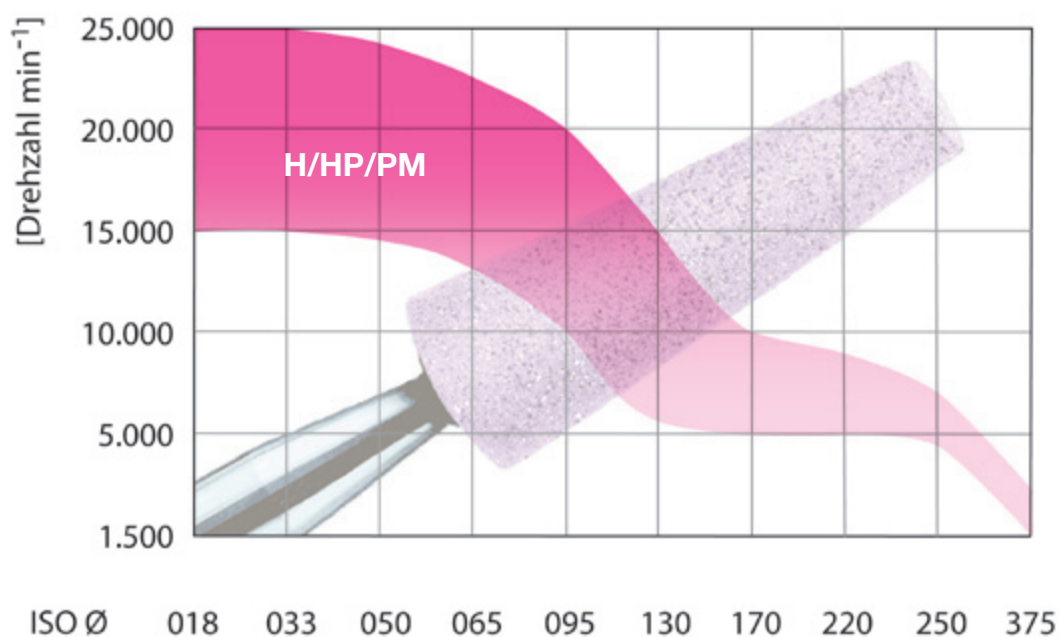
Instrumentensätze, Sonstiges

Sets of instruments, miscellaneous

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Illustrations and photos are protected by copyright. Reproductions, also by extract in any form, are only permitted with written authorization of BUSCH & CO. GmbH & Co. KG Engelskirchen.

To reserve the right to modify our range of products and their design in the sense of progress are traditions of our company.

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Quality management

- ISO 9001:2008
- ISO 13485:2012+AC:2012
- MDD/93/42/EEC A-II/excl. sec4

VDDI
Dental Solutions.
German Manufacturers.