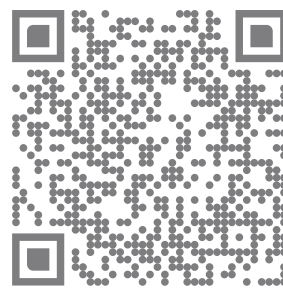
BASIC Catalogue

scan for full version

# BEST OF 2

An exclusive collection

PARTNER  
FOR DENTAL  
INSTRUMENTS

*Solutions for Experts*

 Liebe Leserin, lieber Leser,

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 Dear reader,

Innovation emerges where experience meets new ideas. In our Prime Selection, you will find instruments that have been further developed or completely redesigned with a special focus on modern treatment methods – always with the goal of making your daily work easier.

Whether it's proven classics in a new design or groundbreaking innovations – each of these instruments offers well-thought-out improvements, innovative details, or specialized functions that make your work even more efficient. Developed with dental professionals, made for everyday practice.

I wish you great enjoyment with our Prime Selection.

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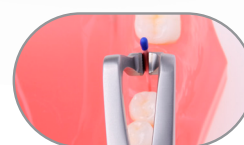
This universal pliers is ideally suitable for removing/extracting and introducing objects such as matrices, interdental wedges, cannulas and syringes, crowns, nerve broaches, root canal files, rotating instruments and much more.

The flattened jaw ends and integrated hard metal blades ensure objects to be held easily and securely.

The outwards curved limbs of the pliers are a particular advantage. They enable a good view of the pliers jaws and onto the area of use, thus making work easier.

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Spoon curette | particularly suitable for the front tooth area

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Plate curette | for removal of granular material in the posterior side tooth area

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Spherical plugger / Spatula | for retrograde fillings, ball form for compression

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For secure gripping and positioning of posts and gutta-percha points. The fine tips ensure access to the root canal.

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Suitable for gripping round or square objects of any kind. For example, root canal instruments, crowns or rotary instruments and for the safe removal of covers (from syringes, canulas).

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The slender tips of the forceps allow a safe grasping of small root/bone splinters and fractured endodontic instruments. To get still better grip, the working tips are finished with a diamond coating.

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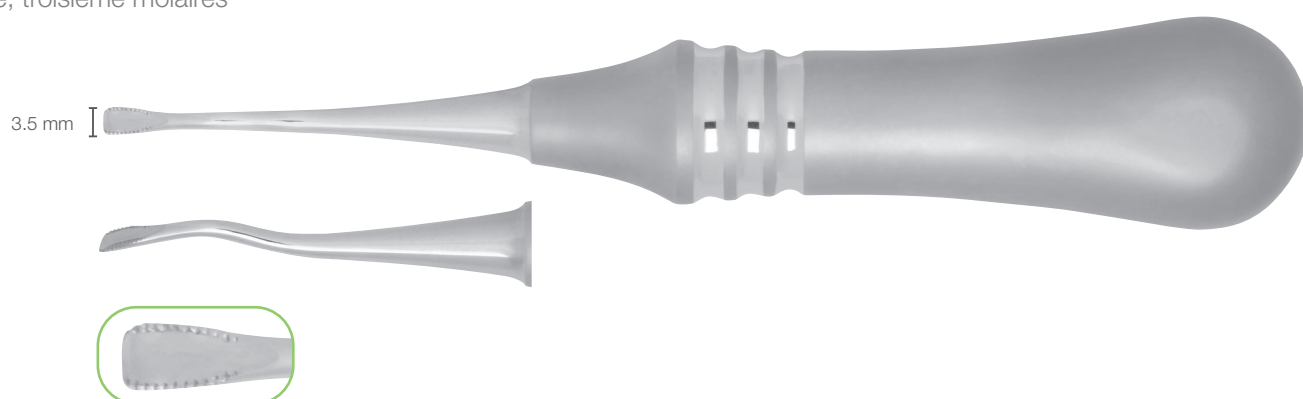
For the extraction and levering of third molars. The serration  
avoids slipping at the teeth.

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## GES LINE | GENTLE EXTRACT SYSTEM

GES LINE | GENTLE EXTRACT SYSTEM  
GES LINE | GENTLE EXTRACT SYSTEM

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GES-LINE dental forceps are specially designed to ensure an optimal fit to the tooth, minimizing the force required during use. The beak runs parallel to the cervical margin, enabling a gentle and atraumatic extraction. The handle ensures a constant force transmission during rotational, compressive, and pulling movements. The F-LINE PLAIN SURFACE design, with its smooth handle, prevents dirt buildup and is easy to clean. Its sandblasted surface reduces distracting light reflections during work.

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Starting extraction with losing the ligaments by using a luxator (here dev-lux instrument). Then wait for bleeding.

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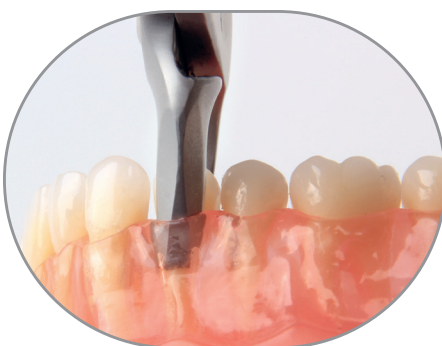
Insert the jaws deeply in the alveole, to get a good grip with the tips of the forceps on the root of the tooth.

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Extract the tooth with small and gentle luxation movements.

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
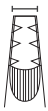
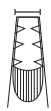
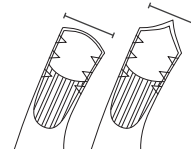
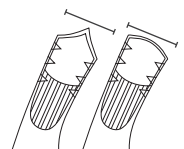
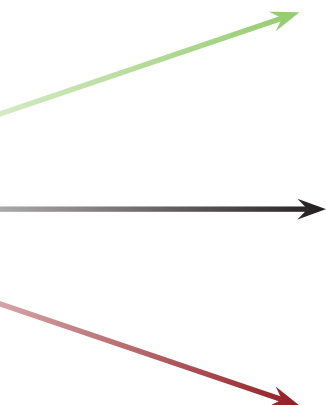
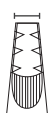
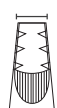
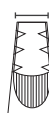
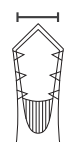
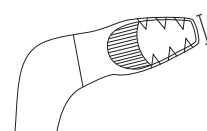
1300-67A FS

## PINK SET FOR MICRO TUNNELING | NACH DR. BERNOTTI

PINK SET FOR MICRO TUNNELING | IN ACCORDANCE TO DR. BERNOTTI

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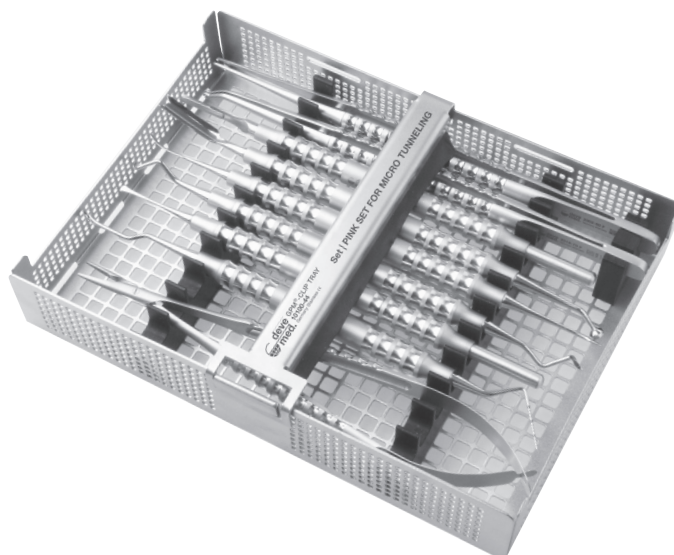The lance-shaped end of the work-piece enables the detachment of the papilla as well as of the marginal gingiva crest. With the slightly curved working part the gingiva is supraparietally removed/loosened - the rounding of the working part prevents the mucosal flap from being pierced.

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Tissue graft packer for inserting the mucosal graft or membrane. The serrated granular packer can be used to condense bone material („sticky bone“).

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The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels. The spatula-shaped working part can be used to attach bone material etc. and at the same time to retract the gingiva.

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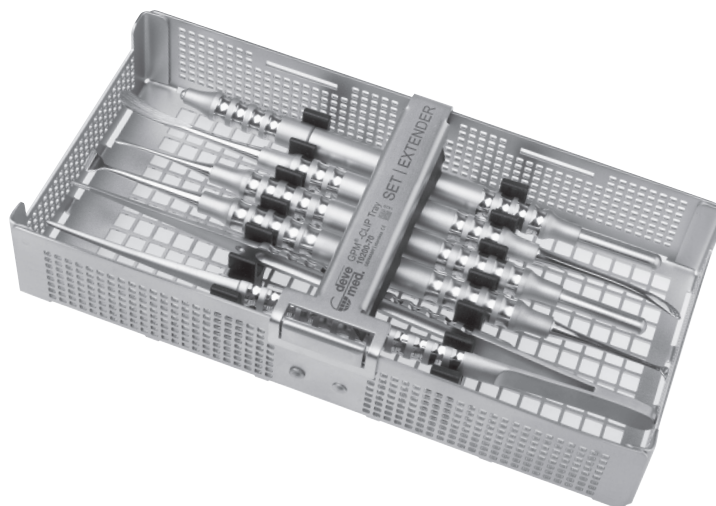
The workpiece ends are arrow-shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces. To avoid injury to the gingiva, the side of the working part which faces the gum is blunt.

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EXTENDER SET | IN ACCORDANCE TO DR. HENRIETTE LERNER

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The periosteum can be loosened and expanded by short, pulling movements exerted with slight pressure. This technique ensures an optimal blood supply, reduces swelling as well as pain and minimizes injuries to the vessels.

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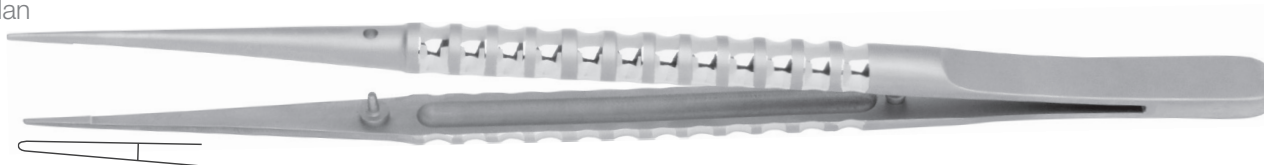
The flat tweezer is intended to held a flap passively for suturing. If there is any tension on the flap the tissue will slip away. Therefore the risk of damaging the structure of the soft tissue and impairment of vascularization is minimized.

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Both cutting edges are smooth. Preferred for sutures & membranes.

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Super-Cut- The razor sharp cutting edge cuts effortlessly and softly through even thicker material. The serrated cutting edge on the second blade prevents slipping from tissue and sutures.

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Tungsten-Carbide - Tungsten-carbide is incorporated into the blades which enables a precise and gentle incision, as well as a long lifespan.

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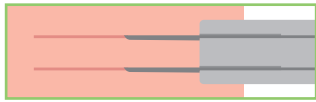
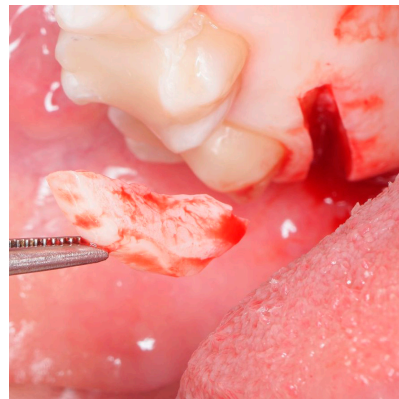
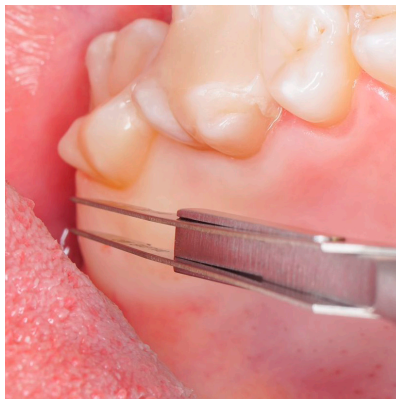
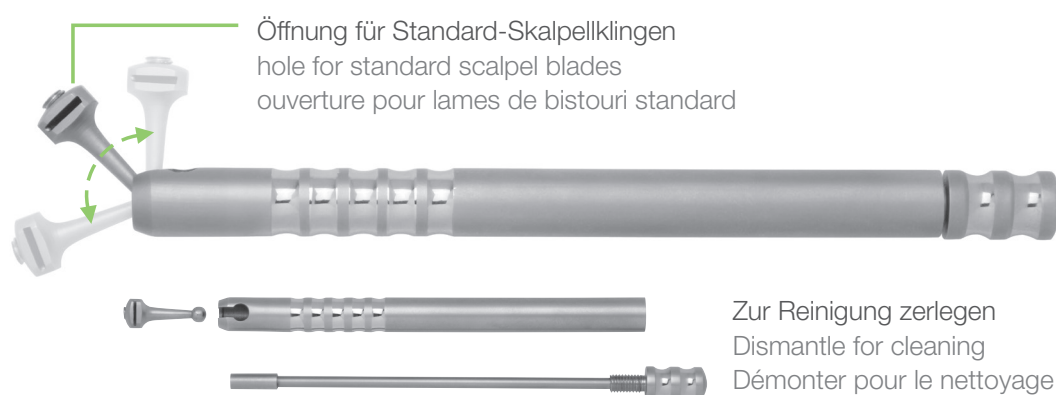
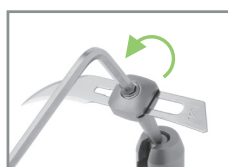
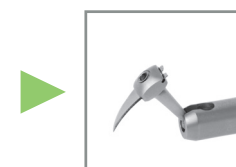
Thanks to the 3D swivel head, the blade can be rotated in almost any direction and positioned according to requirements. This allows for better accessibility to all areas of the mouth, especially in hard-to-reach regions.

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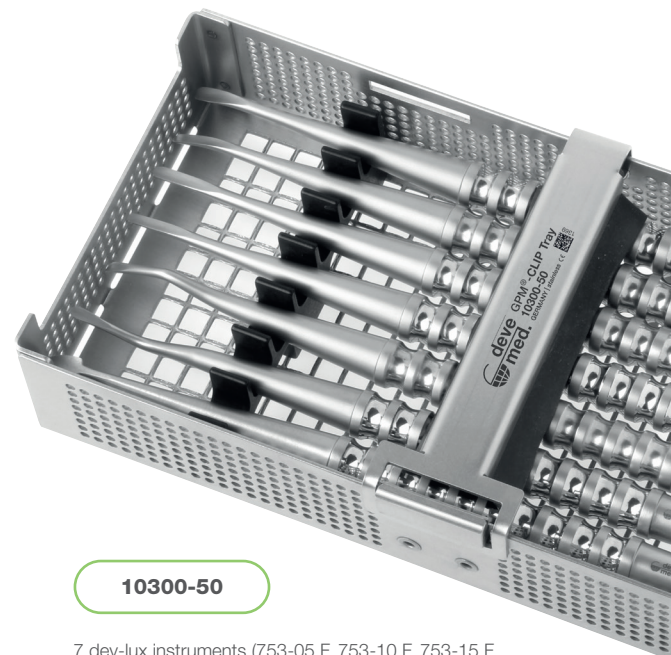
The delicate working parts of the raspatories enable even more minimally invasive atraumatic work.

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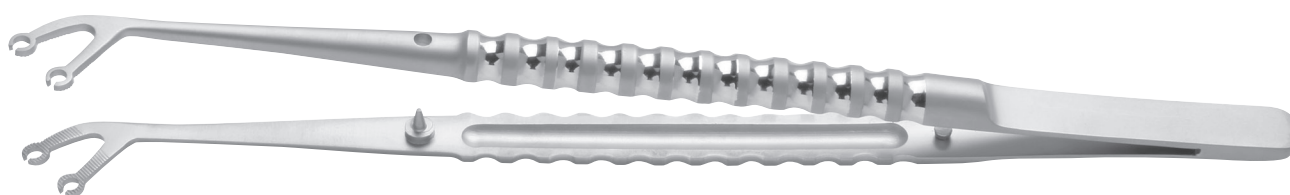
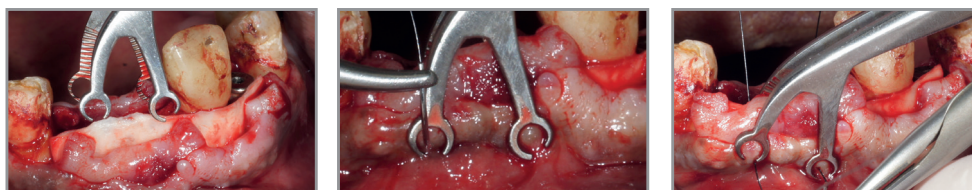
Dr. Bassam Rabie recommends: This forceps serves perfect for fixation and suturing. The shaping of the double tweezers allows a safe reposition of the soft tissue graft. The circular tips avoid tearing of the tissue.

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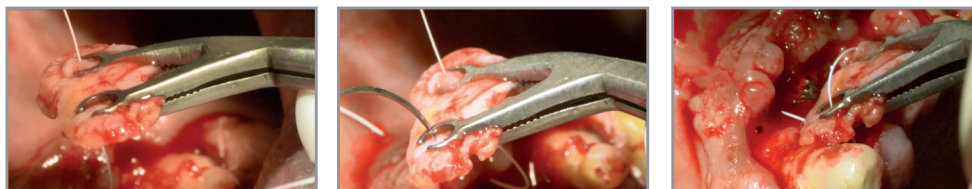
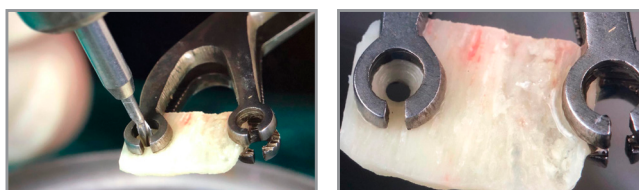
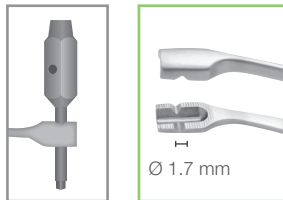
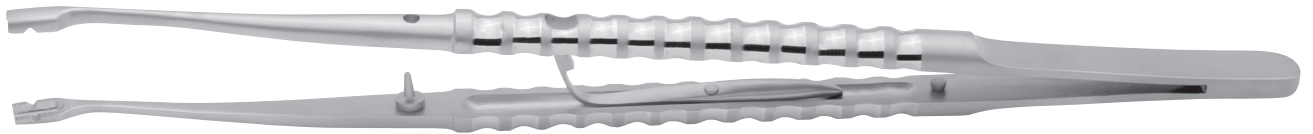
HOLDING FORCEPS FOR SCREW DRIVERS | ACC. TO DR. BARTH

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These holding forceps enable you to work with all major implant system screwdrivers. The abutments can be easily replaced when working with the holding forceps.

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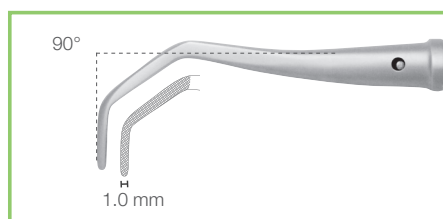
The 90° angle makes these universal forceps unique. For example, the double angulation facilitates the application of matrices or root posts and can be used for restraining soft tissue structures. In addition, it is extremely well suited for apicoectomies or suturing. The cross serrations offer excellent grip for holding objects.

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The serration enables better gripping and loosening of bone residues that can otherwise only be removed with difficulty. Great for removing cysts. Furthermore, it causes a slight bleeding with result of a better healing.

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MICRO NEEDLE HOLDERS WITH LOCK | MINIMALLY INVASIVE SURGERY

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**Now even better:** The entire range of needle holders has been optimized. The improved mechanism ensures smooth and even closure, allowing for comfortable handling during the procedure.

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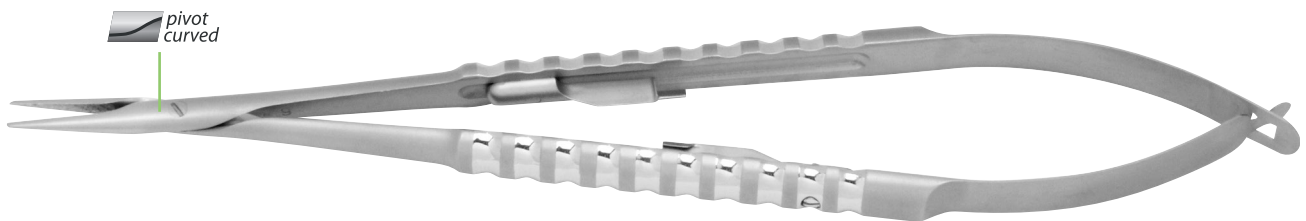
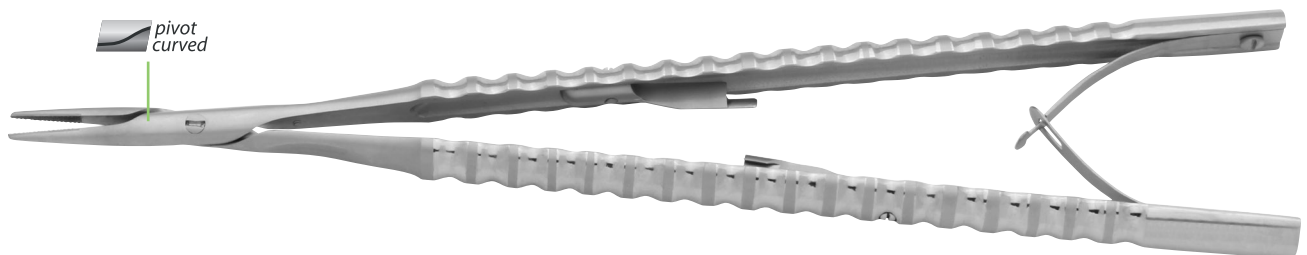
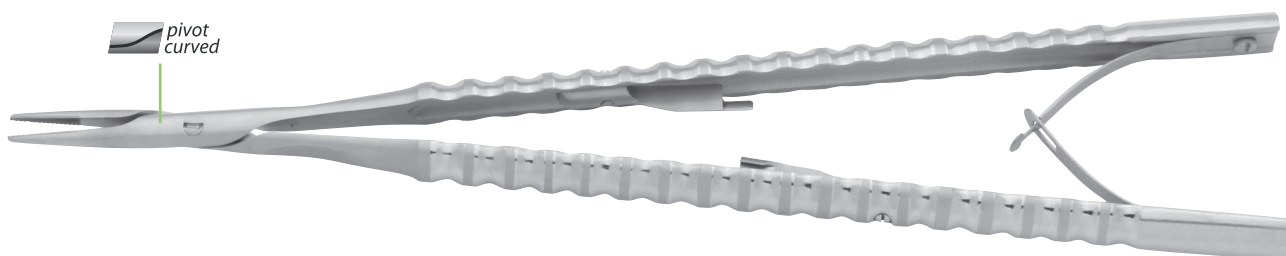
The TiN Perioluxes can be easily inserted along the root surface and cut the periodontal ligament prior to extraction (Caution: no levering). They feature a long, slim profile and sharp blades, minimising tissue trauma and preserving the alveolar bone. The TiN coating improves gliding in the periodontal gap due to its lower coefficient of friction and improves the durability of the blades.

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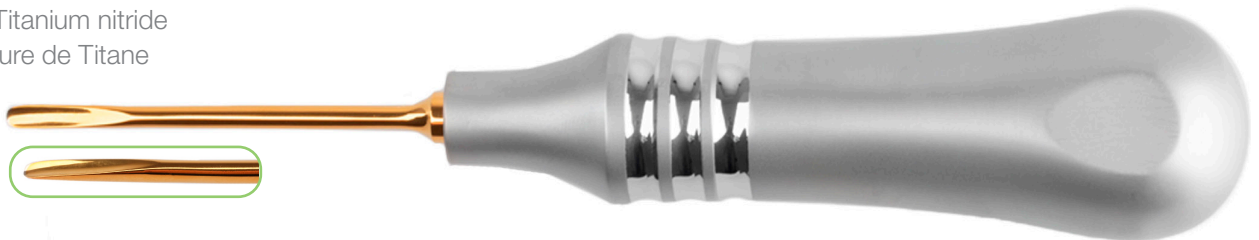
Atraumatic tunneling with minimal incisions for safe flap preparation. Protects delicate tissue and facilitates access, especially in distal areas. Designed by Carole Leconte.

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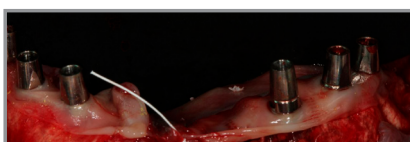
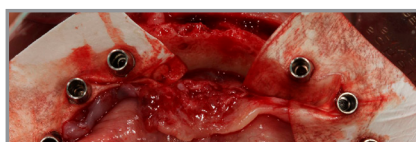
The punch developed by Dr. Henriette Lerner is used for the targeted perforation of membranes (3 mm diameter). It is used during immediate implantation to position membranes over abutments.

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„Tatum“ #4 | 191 mm

## ZYLINDER-AMPULLEN-SPRITZEN | EDELSTAHL

CARTRIDGE SYRINGES | STAINLESS STEEL

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The instrument is used to reach the sinus floor. The back is used to reach the medial wall.

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For membrane detachment in narrow areas. Both ends are sharp and enable tactile sinus elevation.

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
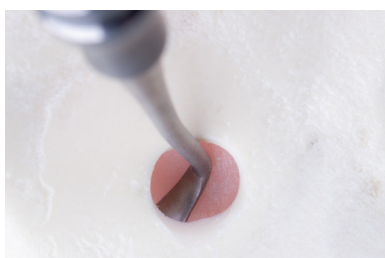
To loosen the membrane and to at-  
tach bone replacement material in  
the maxillary sinus.

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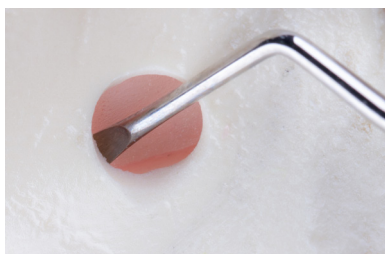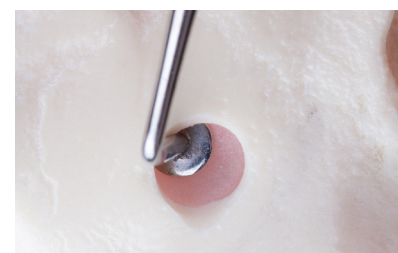
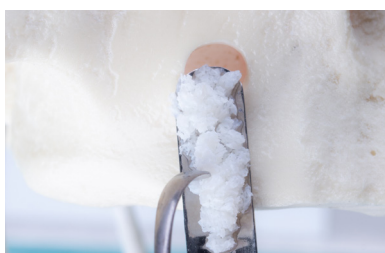
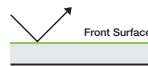
The micro mirrors „BLACKedition“ are adaptable.

Les micro miroirs „BLACKedition“ sont adaptable.

## MICRO SET NACH DR. RABIE & DR. DE LA ROSA

MICRO SET ACC. TO DR. RABIE & DR. DE LA ROSA  
MICRO SET SELON DR. RABIE & DR. DE LA ROSA

**BLACK edition**

Set 10111-50

**998-25 FBC**

F-line

NEW

University of Cawood-Minnesota mod. | 150 mm

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The black coating and cutouts ensure optimal visibility and improved access to hard-to-reach areas. Reflection-free, scratch-resistant surface for high contrast. The smooth surface is particularly hygienic and gentle on mucous membranes. Ideal for surgical procedures, restorative treatments, and intraoral scans.

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The screw cap allows a simple and safe insertion and removal of the micro scalpel blade. Light, rotatable and handy for precise circular incisions.

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These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimally invasive surgery. The scissors also offer the possibility of a rotary movement and are ideally suited for working with magnifying glasses or a microscope.

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Atraumatic micro-forceps offer a compromise between the surgical forceps, which provide a strong grip yet damage tissue, and the less effective anatomical forceps, which however allow a gentle grip. The fine micro teeth prevent perforation and slippage of the tissue even at very slight pressure, thus enabling gentler gripping.

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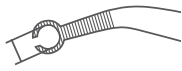
These suture forceps allows an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously. While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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Micro needle holders with lock, with cross serration

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Based on the 1086-21 F model, the micro needle holder additionally features a black DLC coating. This provides an extremely hard, scratch-resistant surface, reduces friction, and facilitates cleaning. The anti-glare coating eliminates distracting reflections, making it ideal for work with magnifying loupes. Additionally, the dark color enhances contrast in the surgical field and ensures precise visibility conditions.

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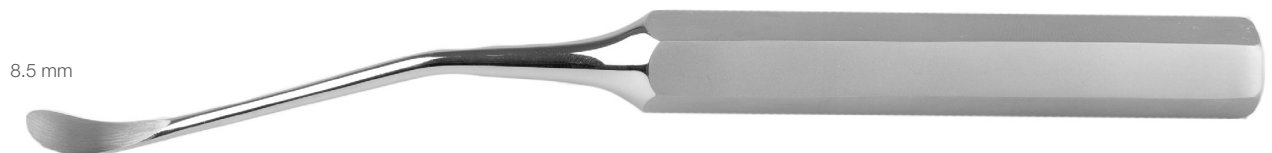
## ZYGOMA INSTRUMENTE

ZYGOMA INSTRUMENTS  
ZYGOMA INSTRUMENTS

### ZYG 1010 H

164 mm

ZAGA # 1 | Zygoma Dissektor  
ZAGA # 1 | Zygoma Dissector  
ZAGA # 1 | Dissecteur de zygoma

### ZYG 1020

184 mm

ZAGA # 2 | Zygoma Retraktor  
ZAGA # 2 | Zygoma Retractor  
ZAGA # 2 | Rétracteur de zygoma

## ZYGOMA INSTRUMENTE

ZYGOMA INSTRUMENTS  
ZYGOMA INSTRUMENTS

### ZYG 1030 H

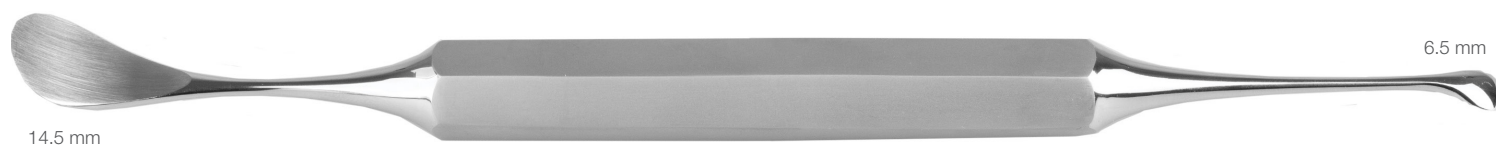
NEW

200 mm

ZAGA # 3 | Maxilla Dissektor

ZAGA # 3 | Maxilla Dissector

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### ZYG 1040 H

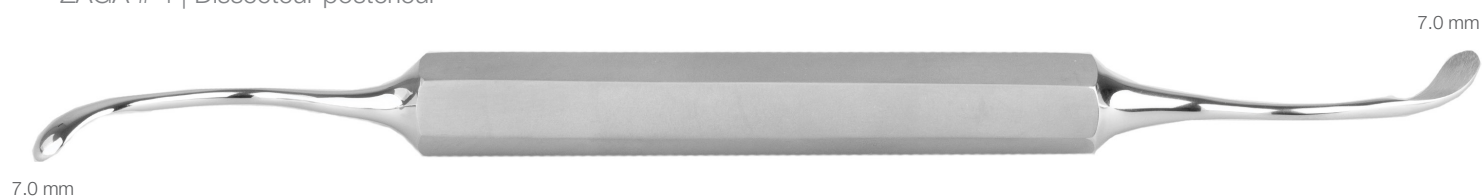
NEW

194 mm

ZAGA # 4 | Posterior Dissektor

ZAGA # 4 | Posterior Dissector

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### ZYG 1050 H

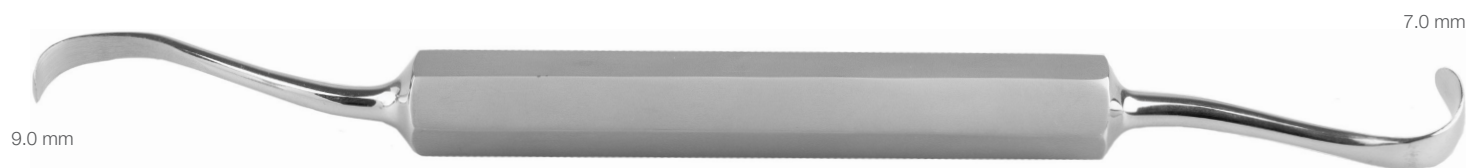
NEW

190 mm

ZAGA # 5 | Nasal Protektor

ZAGA # 5 | Nasal protector

ZAGA # 5 | Protecteur nasal

## ZYGOMA INSTRUMENTE

ZYGOMA INSTRUMENTS  
ZYGOMA INSTRUMENTS

### ZYG 1060

ZYGO # 6 | Zygoma Tiefenmesser  
ZYGO # 6 | Zygoma Depth gauge  
ZYGO # 6 | Zygoma Jauge de profondeur

## ZYGOMA SET - ACC. ZAGA CONCEPT

ZYGOMA SET - ACC. ZAGA CONCEPT  
ZYGOMA SET - ACC. ZAGA CONCEPT

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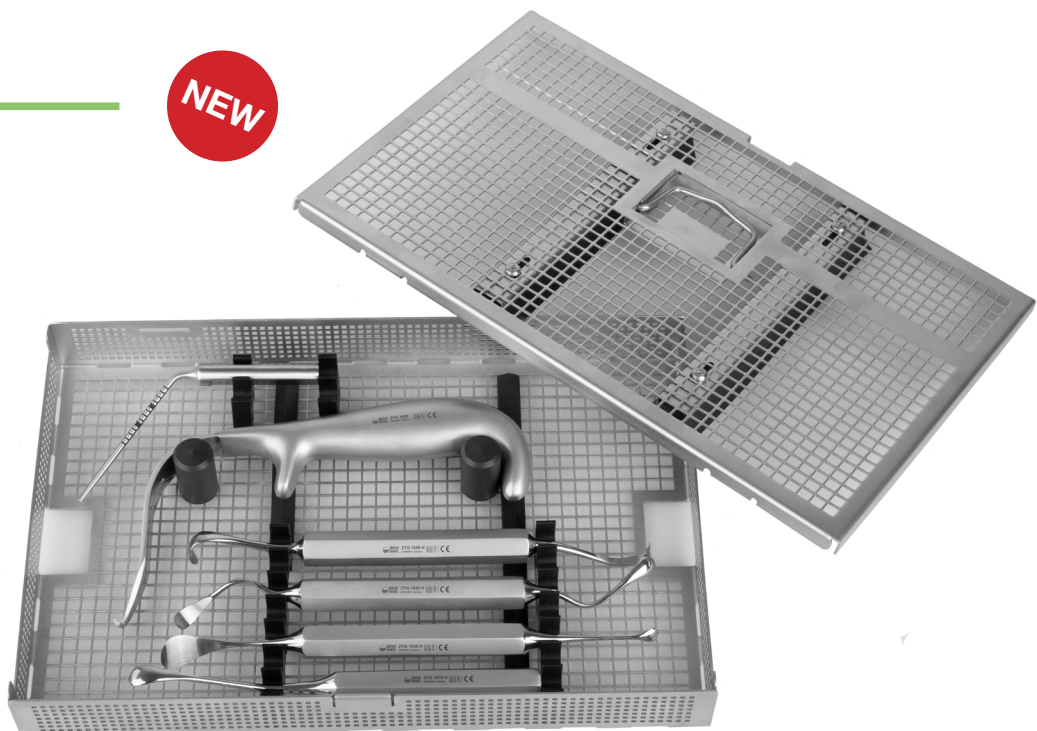
Made of solid stainless steel, ideal for driving pins. Its well-balanced weight distribution ensures a comfortable grip and allows for controlled force transmission.

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devemed GmbH